

The SEGUFIX[®]-System

- Humane and reliable restraint with many positioning options -

SEGUFIX[®] - The Safe Way



SEGUFIX[®]-
Twist Lock



Catalogue 2013



The History of SEGUFIX®

An ancient problem

Since ancient times the necessity of immobilizing a patient for his recovery was common. However, restraining limbs often delayed recovery, for example by causing panic attacks or bedsores. Up to the 1960s restraint methods remained the same. Then the equally simple and brilliant idea of a worried and caring father should open a new chapter in the history of restraints.

A toddler's misery

Said father had a two-year-old son, who suffered from severe pneumonia. When the boy resisted the necessary bed rest, the doctors had no other choice but to tie him down by his hands and feet. Confused and terrified the little boy refused to eat. When faced with this new problem, the doctors ordered force-feeding. The desperate parents couldn't bear that thought and took their child home at their own risk.

Love is inventive

In the comfort of his own home the little boy quickly regained his old spirits and appetite, but he still would not stay in bed. This prompted the father to create a harness to keep his son in bed, but with plenty of freedom to move, foremost leaving hands and feet unrestrained. The sick youngster accepted the harness, stayed in bed and soon recovered.

A solution for many

Years later, the father learned about the extent of medical restraints. Many patients had to suffer, just because they were not staying in bed or simply would fall out of bed - a problem he had already solved for his son many years before. Renowned physicians recognized the benefits for patients and nursing staff. Prof. Dr. Dr. h.c. Bürger-Prinz introduced the new restraint at the University Hospital Hamburg-Eppendorf (UKE). Among experts the SEGUFIX®-System was met with great approval, and shortly after the SEGUFIX®-System was established internationally as the approved method of patient restraint.

Maximum freedom with minimal restraint

Restraint requirements are complex, and at times it may be necessary to restrain limbs. To meet the individual requirements our product range was expanded in close cooperation with medical professionals. Whenever new products are being developed the guiding principle 'Maximum freedom with minimal restraint' is always applied.

The humane restraint

Accordingly the SEGUFIX®-System offers sophisticated positioning and restraint solutions for patients and care personnel. For the first time these solutions allowed for targeted restraint and thus prevent degrading and excessive restraint of past days. Consequently medical experts described the SEGUFIX®-System unanimously as the 'humane restraint' when it was introduced. In all areas of patient care and even in difficult care situations, the humane approach results in great relief for everyone involved.

Reliability in daily care

As inventor and manufacturer of the SEGUFIX®-System we are happy to offer products that allow you to provide safe and reliable care.

As a family business with 40 years of experience, the next generation will also support you as a reliable partner with continuity, quality and professionalism.

The Sánchez-Wysozki family

Product groups

Product group 1000 - SEGUFIX®-Locks and Keys



Product group 2000 - SEGUFIX®-Bed Restraint System



Product group 3000 - SEGUFIX®-Auxiliary Belts



Product group 4000 - SEGUFIX®-Quick Restraint



Product group 5000 - SEGUFIX®-Mobile Restraint



Product group 6000 - SEGUFIX®-Transport System



Product group 7000 - SEGUFIX®-Chair Belt System



Product group 8000 - SEGUFIX®-Accessories: Sheepskin and other helpful products



Locks & Keys
Item no. 1106 - 1215

Bed Restraint System
Item no. 2203 - 2250

Auxiliary Belts
Item no. 3110 - 3245

Quick Restraint
Item no. 4213 - 4235

Mobile Restraint
Item no. 5204 - 5245

Transport System
Item no. 6040 - 6276

Chair Belt System
Item no. 7240 - 7280

More Products
Item no. 8010 - 8022

Table of Contents

General information		Page
The History of SEGUFIX®		2
Overview of the eight SEGUFIX®-Product groups 1000 - 8000		3
Table of Contents		4 - 5
SEGUFIX®-Traffic Light Labeling		6 - 7
Safety Information		8 - 9
Product Information/Instructions for Care		10
Abbreviations for various versions		11

SEGUFIX®-Locks and Keys

Item no.	Products 1000	Page
1206	• SEGUFIX®-Lock	12
1208	• SEGUFIX®-Magnetic Key	12
1209	• SEGUFIX®-Magnetic Key with key ring	12
1210	• SEGUFIX®-Twist Lock	13
1212	• SEGUFIX®-Magnetic Twist Key	13
1213	• Push Lock	13
1215	• Push Lock Key	13

SEGUFIX®-Bed Restraint System

Item no.	Products 2000	Page
2203	• SEGUFIX®-Shoulder Restraint	19
2204	• SEGUFIX®-Hand Restraint	20
2205, 2205/v	• SEGUFIX®-Foot Restraint	21
2211, 2211/v	• SEGUFIX®-Thigh Restraint	22
2212	• SEGUFIX®-Shoulder Additional Belt	19
2217	• SEGUFIX®-Head Positioning Belt	23
2221, 2221/v	• SEGUFIX®-Standard with Crotch Strap	14 - 15
2222	• SEGUFIX®-Complete with Crotch Strap	16
2224, 2224/v	• SEGUFIX®-Hand Restraint lengthend	20
2231, 2231/v	• SEGUFIX®-Standard with Thigh Straps	17 - 18
2232	• SEGUFIX®-Complete with Thigh Straps	18
2250/r	• SEGUFIX®-Compact	23

SEGUFIX®-Auxiliary Belt

Item no.	Products 3000	Page
3110/r	• SEGUFIX®-Brace Strap for legs	25
3240 - 3245	• SEGUFIX®-Extension Belts	24 - 25

SEGUFIX®-Quick Restraint

Item no.	Products 4000	Page
4213/r	• SEGUFIX®-Quick Restraint Shoulder	26
4214/r	• SEGUFIX®-Quick Restraint Hand	26
4215/r	• SEGUFIX®-Quick Restraint Foot	26
4233/r	• SEGUFIX®-Quick Restraint Shoulder	27
4234/r	• SEGUFIX®-Quick Restraint Hand	27
4235/r	• SEGUFIX®-Quick Restraint Foot	27

SEGUFIX®-Mobile Restraint

Item no.	Products 5000	Page
5204/r	• SEGUFIX®-Hand Waist Restraint	31
5218, 5218/v	• SEGUFIX®-Restraining Aid	30
5243	• SEGUFIX®-Stretcher Restraint Shoulder	28
5245, 5245/v	• SEGUFIX®-Stretcher Restraint Foot	28 - 29

SEGUFIX®-Transport System

Item no.	Products 6000	Page
6040	• SEGUFIX®-Crotch Pad for the Support Belt	33
6210/v, 6216/v	• SEGUFIX®-Transport Waist Belt	32
6220	• SEGUFIX®-Suspender Belt	33
6230/v	• SEGUFIX®-Backrest Strap	32
6240, 6244	• SEGUFIX®-Support Belt	33
6250/v, 6256/v	• SEGUFIX®-Multi Belt	34
6260, 6266	• SEGUFIX®-Pelvic Belt	35
6276	• SEGUFIX®-Pelvic Belt with Crotch Strap	35

SEGUFIX®-Chair Belt System

Item no.	Products 7000	Page
7240, 7240/v, 7246	• SEGUFIX®-Chair Belt	36
7250, 7256	• SEGUFIX®-Chair Belt with Crotch Strap	37
7260, 7266	• SEGUFIX®-Chair Belt with Thigh Straps	38
7270/v	• SEGUFIX®-Transport Hand Strap	39
7280, 7280/v	• SEGUFIX®-Transport Foot Strap	39

SEGUFIX®-Accessories

Item no.	Products 8000	Page
8010, 8011	• SEGUFIX®-Sheepskin Products	40
8012	• SEGUFIX®-Sheepskin Padding for SEGUFIX®-Standard with Crotch Strap and Standard with Thigh Straps	40
8013	• SEGUFIX®-Protective Cover	41
8014	• SEGUFIX®-Transport Bag	42
8015	• SEGUFIX®-Response Bag	42
8016	• SEGUFIX®-Pull-up Line	41
8017	• Anti-Slide-Mat	41
8019/GB	• SEGUFIX®-Manual	42
8022	• SEGUFIX®-Laundry Net	42


SEGUFIX®-Component Parts

Item no.	Products 1000 - 8000	Page
1106 - 1114	• SEGUFIX®-Locks and Keys	43
2005 - 2150/re	• SEGUFIX®-Bed Restraint System	43 - 46
3140 - 3110/r	• SEGUFIX®-Auxiliary Belts	47
4113/r - 4115/re	• SEGUFIX®-Quick Restraint	47
5101/r - 5145	• SEGUFIX®-Stretcher Restraint / Mobile Restraint	48
6040 - 6176	• SEGUFIX®-Transport System	48
7140 - 7180	• SEGUFIX®-Chair Belt System	49

Miscellaneous

	Page
• General Terms and Conditions	50
• Purchase Order Form	51

Colour labelling for easy product identification

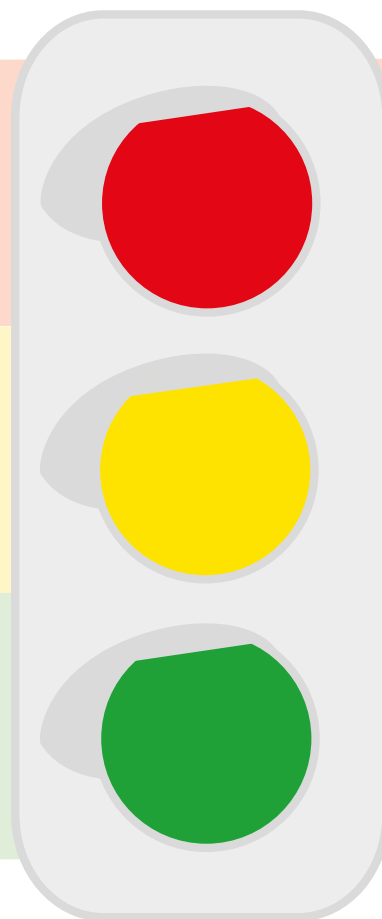

RED → Shoulder and Head

SEGUFIX®-Shoulder Restraint
 SEGUFIX®-Shoulder Additional Belt
 SEGUFIX®-Head Positioning Belt
 SEGUFIX®-Quick Restraint Shoulder
 SEGUFIX®-Stretchers Restraint Shoulder


YELLOW → Waist and Hand

SEGUFIX®-Standard with Crotch Strap
 SEGUFIX®-Standard with Thigh Straps
 SEGUFIX®-Hand Restraint
 SEGUFIX®-Quick Restraint Hand
 SEGUFIX®-Compact
 SEGUFIX®-Hand Restraint Belt

GREEN → Thigh and Foot

SEGUFIX®-Thigh Restraint
 SEGUFIX®-Foot Restraint
 SEGUFIX®-Connecting Belt
 SEGUFIX®-Quick Restraint Foot
 SEGUFIX®-Stretchers Restraint Foot
 SEGUFIX®-Brace Strap for legs

SEGUFIX®-Traffic Light Labelling

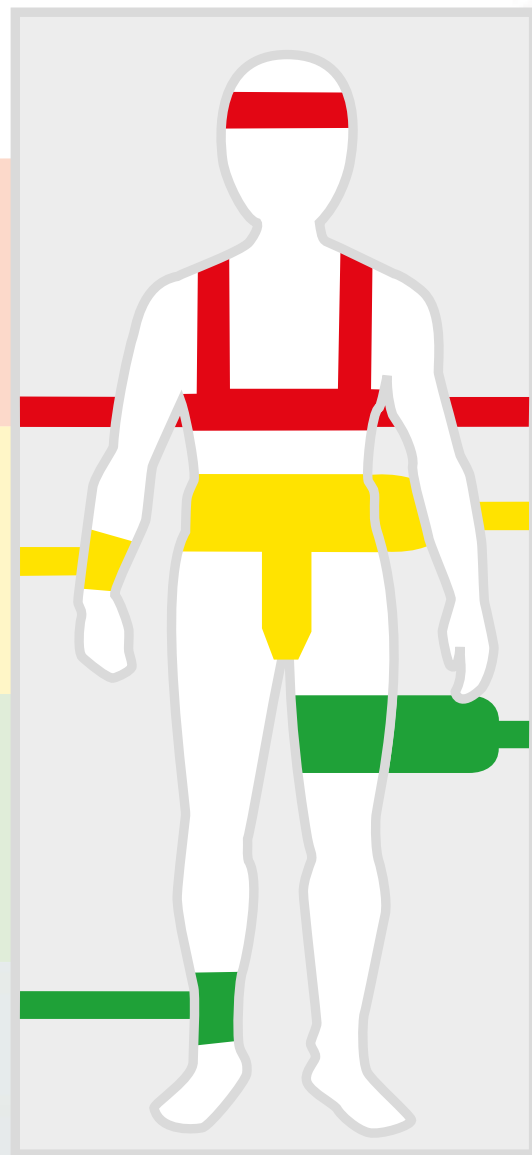


BLUE → Auxiliary Belts

SEGUFIX®-Extension Belts

WHITE → Transport Products

SEGUFIX®-Chair Belt
 SEGUFIX®-Support Belt
 SEGUFIX®-Pelvic Belt
 SEGUFIX®-Multi Belt
 SEGUFIX®-Transport Waist Belt
 SEGUFIX®-Backrest Strap
 SEGUFIX®-Suspender Belt

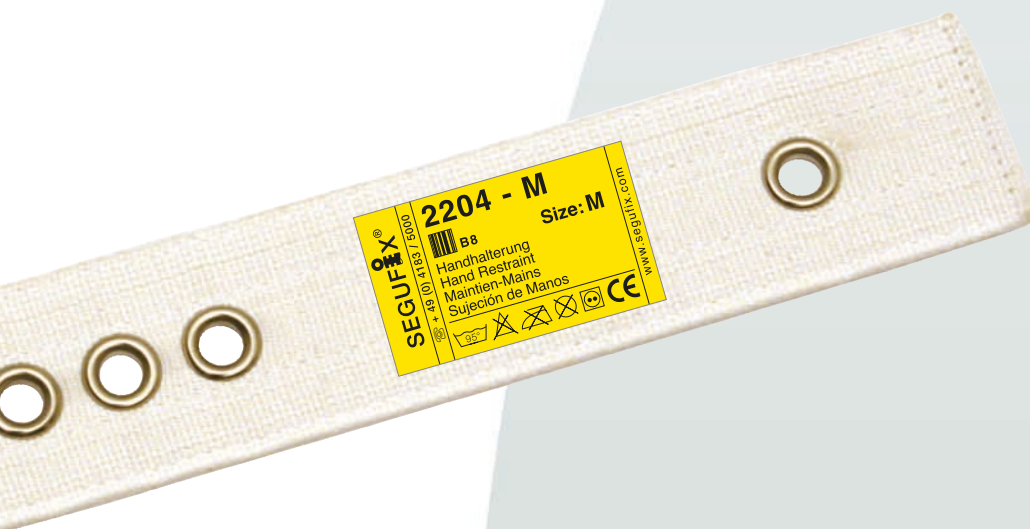

Item no. 2203 - 2250

- Item no. 3110 - 3245

The following instructions for use are only applicable to original SEGUFIX® products. Do not use the SEGUFIX®-Restraint and Positioning System until you read and understand the instructions for use. Failure to use this product properly can cause serious injury or death. Your patient's safety depends on your using the SEGUFIX® System correctly.

SEGUFIX® - Patient restraint according to indications.

1. Requirements for use:

The instructions, legal requirements and physician's orders are to be followed. SEGUFIX® products may only be used by **trained and qualified** nursing staff/private caretakers that are familiar with the proper use of the product. The patient's health must not contraindicate the use of the product.

2. Intended use:

SEGUFIX® products may only be used for the medical purposes, which they were designed for.

3. Remove dangerous objects:

Remove all objects from the reach of the patient, which could lead to injury or may damage the belts (e.g. glasses, jewelry and sharp objects).

4. Inspect products before use:



Inspect before each use for broken stitches, torn, worn out or frayed parts and damaged eyelets. Damaged products could cause injury. Do not use damaged or worn out products.

5. Test closure-systems prior to each use:



Check SEGUFIX®-Locks, Push Locks, Buckle Locks, Snap Locks and Velcro® fasteners for a secure hold. Do not submerge Locks in any type of liquid. This may cause malfunction. If damaged Locks fail to open, straps can be cut with scissors or belt cutters. Check the head of the metal pin for signs of wear and tear (rounding off or compression).

6. Locks may not be under tension:

To open Locks, the black Button must move freely. Therefore use a maximum of four belt layers per Lock (Exception: Products with "r" and "f" in the item number are thicker. These products allow for a maximum of three belt layers per Lock).

7. Pacemaker Warning:



A distance of 10 cm (4 inches) must be maintained between a patient's pacemaker and the SEGUFIX®-Magnetic Key, otherwise a tachycardia might be triggered (alternatively use Push Locks). For possible interference of strong magnets with other patient implants please refer to manufacturer's instructions of these products.


8. Test products for correct attachment:

Frequently check products for correct attachment. The straps must be secured to the frame of the bed (or the folding part of the frame of the bed) keeping them in place tightly on the mattress.

9. Only use original SEGUFIX® products:

For safety reasons SEGUFIX® products must not be altered or be combined with third party products.

10. Use of SEGUFIX® products in vehicles:

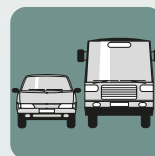

The SEGUFIX®-Transport Belt System does not replace the vehicles seat belt system. SEGUFIX®-Transport Belts may only be used as additional support or for positioning of the patient. Make sure that the patient can be rescued quickly in case of an emergency.

11. Secure but not too tight:

Belts must be fastened tightly, but not interfere with breathing or blood circulation. Loosely fastened products may jeopardize the patient's safety. Frequently check products for correct position.

12. Flammability of SEGUFIX®-Bed Restraints:



According to DIN EN 71-2:03 section 4.3 non flammable. According to DIN 75200 the burning rate is zero. However, products may not come in contact with cigarettes or be exposed to open flame, as this could permanently damage the products and endanger one's health.

13. Proper sizing:

Make sure to choose the proper size according to body and waist sizes (refer to sizing table, next page). Products that are too small or too large compromise the patient's comfort and safety.

14. Expert recommendation (Prof. Dr.-Ing. U. Boenick, TU Berlin):

1. Always use Side Positioning Straps
2. Lock non-parted bedrails in the up position (exception: five-point-restraint)
3. The use of additional restraint elements may be necessary to eliminate bed egress.

Warnings



15. Always attach Side Positioning Straps!

The SEGUFIX®-Standard with Crotch Strap or Thigh Straps must always be used with the Side Positioning Straps attached. Side Positioning Straps prevent the patient from helicoptering in bed or from getting on the edge of the bed causing strangulation, possibly resulting in death. In some cases it is possible that patients get over the edge of the bed even with the Side Positioning Straps attached and the side rails in the up position. In these cases additional restraint measures must be taken.

Important: Insert the Side Positioning Straps through the respective loops (according to instructions) before securing them to the bed belt.

16. Beds, chairs and stretchers:

- Use SEGUFIX® products only on beds, stable chairs and stretchers that allow the product to be attached to, in accordance with the instructions.
- Products must be attached in such manner, that they cannot get out of place.
- Note that straps and moving mechanics of beds, stretchers or chairs (or alternating pressure pads and the like), may interfere with each other causing damage and loss of function.
- The attachment points must be free of sharp edges.

- SEGUFIX® products do not prevent beds, chairs or stretchers from tipping over or from unintentional movement.

Helpful hints:


European standards for beds:

DIN EN 60601-2-52

European standards for wheel chairs:

DIN EN 12182 and DIN EN 12183


17. All side rails must be in the up position:

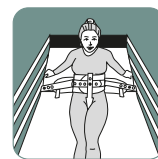

To prevent accidents always lock side rails in the up position (exception: 5-point-restraint).

Note: Do not use with parted side rails except with a gap protector that effectively prevents the patient from slipping through the gap. An open gap could allow the patient to slip through even with the restraint fastened possibly causing strangulation.

18. Monitor patients in restraints:

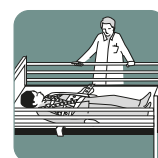

Patients in restraints require frequent monitoring. Aggressive, agitated or restless patients and patients in danger of aspiration require constant monitoring.

Waist Size Guide:

SEGUFIX®-Standard with Crotch Strap or Standard with Thigh Straps

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Instructions for Care

- **SEGUFIX®-Restraints** (white) are washable in hot wash cycles up to 95°C. Lower temperature and laundry nets will prolong product life. Avoid methods that might bend metal parts of the system (e.g. laundry line, dry pressing, rotary irons), eyelets may be damaged.
Not pre-washed, up to 8 % shrinkage. Store in a dry place.
Detergents: non-aggressive, no fabric softener, no bleach.
Dryer: delicate cycle at low temperature.
Disinfection: may be disinfected using chemothermal disinfection washing procedure (item 3.1.2 "catalogue of tested and approved disinfectants and disinfection procedures" published by the Robert-Koch-Institute). Overdosing can damage the product.
- **SEGUFIX®-Transport Belts** (coloured) are washable in warm wash cycles up to 30°C. Avoid methods that might bend metal parts of the system (e.g. laundry line, dry pressing, rotary irons), eyelets may be damaged. Store in a dry place.
Detergents: non-aggressive, no fabric softener, no bleach.
Dryer: delicate cycle at low temperature.
- **SEGUFIX®-Locks and metal pins**, remove all Locks and metal pins before washing or drying. Laundering will damage Locks and possibly cause them to fail. Soiled Locks may be cleaned with a damp cloth. Never submerge Locks in any type of liquid.
- **Velcro® fastener**, fasten before washing and drying to prevent lint buildup. Occasionally use a stiff brush to remove lint from Velcro® fastener. Even with normal use, Velcro® material has the tendency to collect lint. This can reduce the grip strength. Before each use, test grip strength of the Velcro® fastener.
- **Durability** depends on the intensity of use and is prolonged by gentle laundry procedures (detergent/laundry net).
- **SEGUFIX®-Sheepskin products** are easy to clean. Detailed care instructions for medical sheepskins can be found on page 40.



Laundry Net:

The SEGUFIX®-Laundry Net protects the strap material during the wash cycle.

For more information see page 42.

Custom-made products

As developer and inventor of the SEGUFIX®-Systems we gladly design and manufacture products according to your specifications.

Abbreviations for various versions

Basic version



Version f - Fleece

- for sensitive skin
- skin-friendly padding
- pleasant and soft
- 100 % pure Cotton



Version r - reinforced

- eyelet straps reinforced for higher demands




Version e - extended bed belts

- for beds wider than 1 m / 40 inches, extended on each side by approx. 20 cm / 8 inches



Version v – Velcro® fastener / Buckle Lock

- positioning aid
- not a restraint

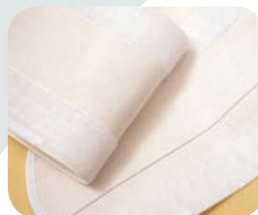

Note:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.



SEGUFIX®-Lock
for secure closing of the
SEGUFIX®-Systems

Item no.	Price/€
1206	6,40



SEGUFIX®-Magnetic Key
for quick release of the
SEGUFIX®-Lock

Item no.	Price/€
1208	4,00



**SEGUFIX®-Magnetic Key
with attachment for key ring**
for quick release of the
SEGUFIX®-Lock

Item no.	Price/€
1209	4,60



SEGUFIX®-Button
black, without SEGUFIX®-Metal Pin

Item no.	Price/€
1106	4,60



SEGUFIX®-Metal Pin
for SEGUFIX®-Button

Item no.	Price/€
1117	1,90

Important:

SEGUFIX®-Locks must not be washed.
Clean locks with damp cloth, if neces-
sary. Do not submerge Locks in any
type of liquid.

Recommended retail price in €
sales tax not included



SEGUFIX®-Twist Lock

for secure closing of the SEGUFIX®-System (compatible with selected SEGUFIX®-products manufactured after 11/2003)

Item no.	Price/€
1210	11,60



SEGUFIX®-Magnetic Twist Key

for quick release of the SEGUFIX®-Twist Lock

Item no.	Price/€
1212	6,40



SEGUFIX®-Twist Button

green, without SEGUFIX®-Square Pin

Item no.	Price/€
1110	6,40



SEGUFIX®-Square Pin

for SEGUFIX®-Twist Button

Item no.	Price/€
1111	5,20



Push Lock

- for closing of the SEGUFIX®-System
- without magnetic mechanism
- suitable for use on pacemaker patients

Item no.	Price/€
1213	19,80



Push Lock Key


for release of the Push Lock

Item no.	Price/€
1215	10,50

SEGUFIX®-Bed Restraint System



Restraint option with
item no. 2221 and 2203
and extra SEGUFIX®-Lock

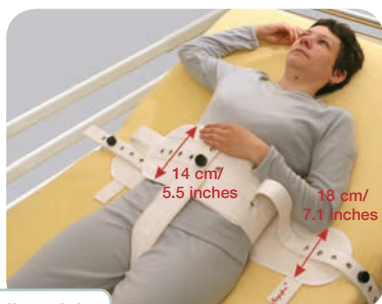

For slim adults




Version f
with fleece

SEGUFIX®-Standard with Crotch Strap and Magnetic Lock

for treatment and prevention of bedsores,
as well as positioning and restraint of
patients

Consisting of:

- 1 SEGUFIX®-Standard with Crotch Strap
(with integrated Side Positioning Straps)
- 3 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

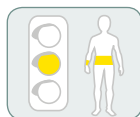


Item no.	Price/€
2221 - M	138,30
2221 - L	149,00
2221/e - M	138,30
2221/e - L	149,00
2221/r - M	154,00
2221/r - L	166,30
2221/r - XL	194,60
2221/re - M	154,00
2221/re - L	166,30
2221/re - XL	194,60
2221/f - M	173,20
2221/f - L	192,70
2221/ef - M	173,20
2221/ef - L	192,70
2221/rf - M	193,45
2221/rf - L	215,85
2221/rf - XL	269,10
2221/ref - M	193,45
2221/ref - L	215,85
2221/ref - XL	269,10

SEGUFIX®-Standard with Crotch Strap and Magnetic Lock

for treatment and prevention of bedsores,
as well as positioning and restraint of
patients

Consisting of:

- 1 SEGUFIX®-Standard with Crotch Strap
(with integrated Side Positioning Straps)
- 3 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

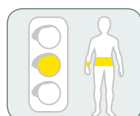

Item no.	Price/€
2221 - S	138,30
2221/e - S	138,30
2221/r - S	154,00
2221/re - S	154,00
2221/f - S	173,20
2221/ef - S	173,20
2221/rf - S	193,45
2221/ref - S	193,45

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Restraint option with
item no. 2221 and 2203
and extra SEGUFIX®-Lock


SEGUFIX®-Standard with Crotch Strap and Magnetic Lock

for use in pediatric bed

Width of waist belt: 14 cm* / 5.5* inches

Consisting of:

- 1 SEGUFIX®-Standard with Crotch Strap
(with integrated Side Positioning Straps)
- 3 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

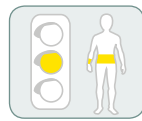

95°

Item no.	Price/€
2221 - XS	132,20
2221/e - XS	132,20
2221/r - XS	146,80
2221/re - XS	146,80
2221/f - XS	166,50
2221/ef - XS	166,50
2221/rf - XS	185,75
2221/ref - XS	185,75



Waist Belt:
Size XS/S: 14 cm / 5.5 inches
Size M/L: 18 cm / 7.1 inches

Bed Belt:
Size XS: 14 cm / 5.5 inches
Size S/M/L: 18 cm / 7.1 inches


SEGUFIX®-Standard Velcro® with Crotch Strap

- positioning aid
- for treatment and prevention of
bedsores
- not a restraint**

Attaches to the bed with metal buckles

Consisting of:

- 1 SEGUFIX®-Standard Velcro® with
Crotch Strap
(with integrated Side Positioning Straps)

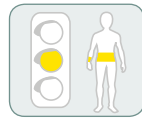

95°

Item no.	Price/€
2221/v - XS	132,20
2221/v - S	138,30
2221/v - M	138,30
2221/v - L	149,00
2221/ev - XS	132,20
2221/ev - S	138,30
2221/ev - M	138,30
2221/ev - L	149,00

**Note Velcro® fastener:

However some jurisdictions might
consider Velcro® products a restraint,
should the patient not be able to
remove them independently.
Please check your applicable legal
requirements.

Waist girth

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Hand Restraint

size*	wrist girth	
XS	12 - 19 cm	4.7 - 7.5 inches
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Foot Restraint

size*	wrist girth	
XS	17 - 25 cm	6.7 - 9.8 inches
M	22 - 30 cm	8.7 - 11.8 inches
L	27 - 33 cm	10.6 - 13.0 inches

Note:

Various possible restraint applications (from loose to tight). For more information see SEGUFIX®-Manual.



SEGUFIX®-Complete with Crotch Strap and Magnetic Lock

2222 - XS consisting of:	Item no.
1 SEGUFIX®-Standard with Crotch Strap	2221 - XS
1 SEGUFIX®-Shoulder Restraint	2203 - XS
1 SEGUFIX®-Hand Restraint	2204 - XS
1 SEGUFIX®-Foot Restraint	2205 - XS


Size **XS** only for **pediatric beds**

2222 - M consisting of:	Item no.
1 SEGUFIX®-Standard with Crotch Strap	2221 - M
1 SEGUFIX®-Shoulder Restraint	2203 - M
1 SEGUFIX®-Hand Restraint	2204 - M
1 SEGUFIX®-Foot Restraint	2205 - M

2222 - L consisting of:	Item no.
1 SEGUFIX®-Standard with Crotch Strap	2221 - L
1 SEGUFIX®-Shoulder Restraint	2203 - L
1 SEGUFIX®-Hand Restraint	2204 - L
1 SEGUFIX®-Foot Restraint	2205 - L

Included:

- 11 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

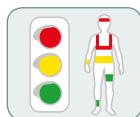

Item no.	Price/€
2222 - XS	255,30
2222 - M	268,40
2222 - L	299,00
2222/e - XS	255,30
2222/e - M	268,40
2222/e - L	299,00
2222/r - XS	286,80
2222/r - M	302,10
2222/r - L	338,10
2222/re - XS	286,80
2222/re - M	302,10
2222/re - L	338,10

SEGUFIX®-Complete with Crotch Strap, item no. 2222

SEGUFIX®-Thigh Restraint, item no. 2211 (page 22)


SEGUFIX®-Shoulder Additional Belt, item no. 2212 (page 19)

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

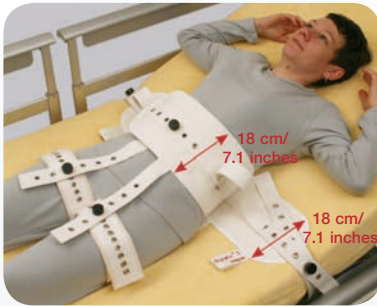

Restraint option with
item no. 2231 and 2203



For slim adults

SEGUFIX®-Thigh Restraint



size*	thigh girth	
S	38 - 52 cm	15.0 - 20.5 inches
M	50 - 72 cm	19.7 - 28.4 inches
L	58 - 88 cm	22.8 - 34.7 inches
XL	85 - 115 cm	33.5 - 45.3 inches

SEGUFIX®-Standard with Thigh Straps and Magnetic Lock

for treatment and prevention of bedsores, as well as positioning and restraint of patients

Consisting of:

- 1 SEGUFIX®-Standard with Thigh Straps (with integrated Side Positioning Straps)
- 5 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

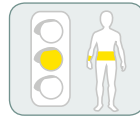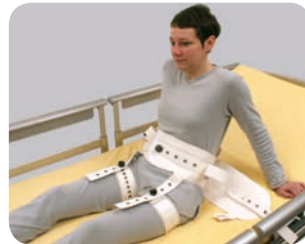

95°

Item no. Price/€

2231 - M 179,40

2231 - L 190,10

2231/e - M 179,40

2231/e - L 190,10

2231/r - M 195,10

2231/r - L 207,40

2231/r - XL 242,70

2231/re - M 195,10

2231/re - L 207,40

2231/re - XL 242,70

2231/f - M 214,30

2231/f - L 233,80

2231/ef - M 214,30

2231/ef - L 233,80

2231/rf - M 234,55

2231/rf - L 256,95

2231/rf - XL 317,20

2231/ref - M 234,55

2231/ref - L 256,95


2231/ref - XL 317,20

SEGUFIX®-Standard with Thigh Straps and Magnetic Lock

for treatment and prevention of bedsores, as well as positioning and restraint of patients

Consisting of:

- 1 SEGUFIX®-Standard with Thigh Straps (with integrated Side Positioning Straps)
- 5 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

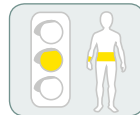

95°

Item no. Price/€

2231 - S 179,40

2231/e - S 179,40

2231/r - S 195,10

2231/re - S 195,10

2231/f - S 214,30

2231/ef - S 214,30

2231/rf - S 234,55

2231/ref - S 234,55

Waist girth

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

SEGUFIX®-Bed Restraint System



Waist Belt:

Size S: 14 cm / 5.5 inches

Size M/L: 18 cm / 7.1 inches

Bed Belt:

Size S/M/L: 18 cm / 7.1 inches

**Note Velcro® fastener:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.


SEGUFIX®-Standard Velcro® with Thigh Straps

- positioning aid
- for treatment and prevention of bedsores
- not a restraint**

Attaches to the bed with metal buckles

Consisting of:

- 1 SEGUFIX®-Standard with Thigh Straps (with integrated Side Positioning Straps)

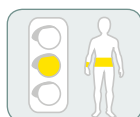

Item no.	Price/€
2231/v - S	179,40
2231/v - M	179,40
2231/v - L	190,10
2231/ev - S	179,40
2231/ev - M	179,40
2231/ev - L	190,10



SEGUFIX®-Thigh Restraint

size*	thigh girth	
M	50 - 72 cm	19.7 - 28.4 inches
L	58 - 88 cm	22.8 - 34.7 inches

SEGUFIX®-Hand Restraint

size*	wrist girth	
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Foot Restraint

size*	ankle girth	
M	22 - 30 cm	8.7 - 11.8 inches
L	27 - 33 cm	10.6 - 13.0 inches


SEGUFIX®-Complete with Thigh Straps and Magnetic Lock

2232 - M consisting of:	Item no.
1 SEGUFIX®-Standard with Thigh Straps	2231 - M
1 SEGUFIX®-Shoulder Restraint	2203 - M
1 SEGUFIX®-Hand Restraint	2204 - M
1 SEGUFIX®-Foot Restraint	2205 - M

2232 - L consisting of:	Item no.
1 SEGUFIX®-Standard with Thigh Straps	2231 - L
1 SEGUFIX®-Shoulder Restraint	2203 - L
1 SEGUFIX®-Hand Restraint	2204 - L
1 SEGUFIX®-Foot Restraint	2205 - L

Included:

- 13 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

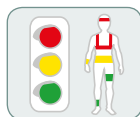

Item no.	Price/€
2232 - M	309,50
2232 - L	340,10
2232/e - M	309,50
2232/e - L	340,10
2232/r - M	343,20
2232/r - L	379,20
2232/re - M	343,20
2232/re - L	379,20

Waist girth

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Size XS and S



Size M and L



Note:

The SEGUFIX®-Shoulder Additional Belt is not suitable for the Shoulder Restraint size XS (children).

SEGUFIX®-Shoulder Restraint with Magnetic Lock


prevents the patient from sliding up out of the SEGUFIX®-Standard

Application:

SEGUFIX®-Standard with Crotch Strap or Standard with Thigh Straps required

Consisting of:

- 1 SEGUFIX®-Shoulder Strap
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

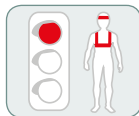

95°

Item no.	Price/€
2203- XS	42,10
2203 - S	45,50
2203 - M	45,50
2203 - L	48,60
2203/r - XS	46,70
2203/r - S	50,60
2203/r - M	50,60
2203/r - L	54,90

SEGUFIX®-Shoulder Additional Belt with Magnetic Lock for the

- Shoulder Restraint
- Quick Restraint Shoulder
- Stretcher Restraint Shoulder

- prevents the patient from sitting up
- requires continuous monitoring


Application:

SEGUFIX®-Standard with Crotch Strap or Standard with Thigh Straps and SEGUFIX®-Shoulder Restraint required

Total length: 114 cm* / 44.9 inches*

Consisting of:

- 2 SEGUFIX®-Shoulder Additional Straps
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

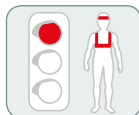

95°

Item no.	Price/€
2212	29,90
2212/r	34,50

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Hand Restraint

size*	wrist girth	
XS	12 - 19 cm	4.7 - 7.5 inches
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Hand Restraint with Magnetic Lock

- adjustable restraint of one or both hands
- prevents patients from injuring themselves or others

Application:


SEGUFIX®-Standard with Crotch Strap or Standard with Thigh Straps required

Consisting of:

- 2 SEGUFIX®-Hand Straps
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

Various possible applications (degree of restraint individually adjustable).

For more information see SEGUFIX®-Manual.

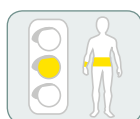

Item no.	Price/€
2204 - XS	29,90
2204 - M	30,90
2204 - L	37,00
2204/r - XS	34,50
2204/r - M	35,70
2204/r - L	42,80



SEGUFIX®-Hand Restraint lengthened


size*	wrist girth	
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Hand Restraint lengthened with Magnetic Lock

- for use without Standard with Crotch Strap or Standard with Thigh Straps
- attaches directly to the bed

Consisting of:

- 2 SEGUFIX®-Hand Straps, lengthened
- 4 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

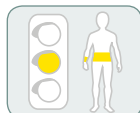

Item no.	Price/€
2224 - M	52,20
2224 - L	62,60
2224/r - M	60,00
2224/r - L	72,00



SEGUFIX®-Hand Restraint lengthened with Velcro® fastener

size*	wrist girth	
M	16 - 24 cm	6.3 - 9.5 inches


SEGUFIX®-Hand Restraint lengthened with Velcro® fastener

- not a restraint**
- for use without Standard with Crotch Strap or Standard with Thigh Straps
- attaches directly to the bed

Attaches to the bed with metal buckles

Consisting of:

- 2 SEGUFIX®-Hand Straps, lengthened

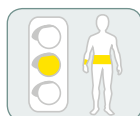

Item no.	Price/€
2224/v - M	39,50



SEGUFIX®-Foot Restraint

size*	ankle girth	
XS	17 - 25 cm	6.7 - 9.8 inches
M	22 - 30 cm	8.7 - 11.8 inches
L	27 - 33 cm	10.6 - 13.0 inches

SEGUFIX®-Foot Restraint with Magnetic Lock


with gradually adjustable movement from loose to tight of one or both feet

Size **XS** only for **pediatric beds**

Versions **e** and **re** with extended Connecting Belt for beds with a width of more than 1 m / 40 inches.

Consisting of:

- 2 SEGUFIX®-Foot Straps
- 1 SEGUFIX®-Connecting Belt
- 4 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

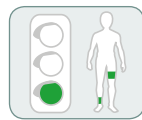

Item no.	Price/€
2205 - XS	51,10
2205 - M	53,70
2205 - L	64,40
2205/e - XS	51,10
2205/e - M	53,70
2205/e - L	64,40
2205/r - XS	58,80
2205/r - M	61,80
2205/r - L	74,10
2205/re - XS	58,80
2205/re - M	61,80
2205/re - L	74,10



SEGUFIX®-Foot Restraint Velcro®

size*	ankle girth	
XS	17 - 25 cm	6.7 - 9.8 inches
M	22 - 30 cm	8.7 - 11.8 inches

SEGUFIX®-Foot Restraint with Velcro® fastener

- not a restraint**
- gradual restraint from loose to tight/positioning of one or both feet


Attaches to the bed with metal buckles

Size **XS** only for **pediatric beds**

Version **ev** with extended Connecting Belt for beds with a width of more than 1 m / 40 inches.

Consisting of:

- 2 SEGUFIX®-Foot Straps
- 1 SEGUFIX®-Connecting Belt

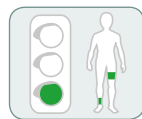

Item no.	Price/€
2205/v - XS	51,10
2205/v - M	53,70
2205/ev - XS	51,10
2205/ev - M	53,70

**Note Velcro® fastener:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.

Waist girth

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Thigh Restraint


size*	thigh girth	
S	38 - 52 cm	15.0 -20.5 inches
M	50 - 72 cm	19.7 -28.4 inches
L	58 - 88 cm	22.8 -34.7 inches

SEGUFIX®-Thigh Restraint with Magnetic Lock

- limits movement of the legs
- optional addition to a 5-point-restraint

Consisting of:

- 1 SEGUFIX®-Thigh Belt
- 4 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

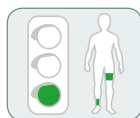

Item no.	Price/€
2211 - S	75,70
2211 - M	75,70
2211 - L	83,30
2211/e - S	75,70
2211/e - M	75,70
2211/e - L	83,30
2211/r - S	87,00
2211/r - M	87,00
2211/r - L	95,70
2211/re - S	87,00
2211/re - M	87,00
2211/re - L	95,70



SEGUFIX®-Thigh Restraint Velcro®

size*	thigh girth	
M	50 - 72 cm	19.7 -28.4 inches
L	58 - 88 cm	22.8 -34.7 inches


SEGUFIX®-Thigh Restraint with Velcro® fastener

- limits movement of the legs
- not a restraint**

Attaches to the bed with metal buckles

Consisting of:


- 1 SEGUFIX®-Thigh Belt

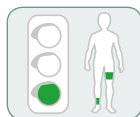

Item no.	Price/€
2211/v - M	75,70
2211/v - L	83,30
2211/ev - M	75,70
2211/ev - L	83,30

**Note Velcro® fastener:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.

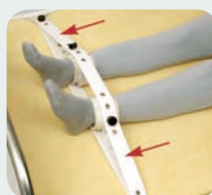

Note:

To further restrict movement, the SEGUFIX®-Brace Strap for legs (item no. 3110/r, page 25) may be used in combination with the SEGUFIX®-Foot Restraint (item no. 2205, page 21), the SEGUFIX®-Quick Restraint Foot (item no. 4215/r and 4235/r, page 26-27) or the SEGUFIX®-Thigh Restraint (item no. 2211, see above).

Waist girth

XS*	S*	M*	L*	XL*
children in pediatric bed	slim adults/ children in adult's bed	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener




SEGUFIX®-Head Positioning Belt with Magnetic Patent Lock/Velcro® fastener

- to position the head
- not a restraint**

Consisting of:

- 1 SEGUFIX®-Bed Belt with Forehead Strap
- 1 SEGUFIX®-Chin Strap
- 1 SEGUFIX®-Positioning Strap
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

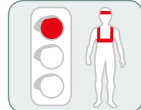


95°

Item no.	Price/€
2217	80,80
2217/e	80,80

Note:

- The SEGUFIX®-Head Positioning Belt is not a restraint, but a positioning aid.
- The SEGUFIX®-Head Positioning Belt was designed to hold the patient's head in the desired position.
- Only for use in connection with other restraints, to minimize the risk of neck or spinal injury.
- The use of the SEGUFIX®-Head Positioning Belt requires continuous monitoring, especially because involuntary or voluntary movements may cause neck or spinal injuries.

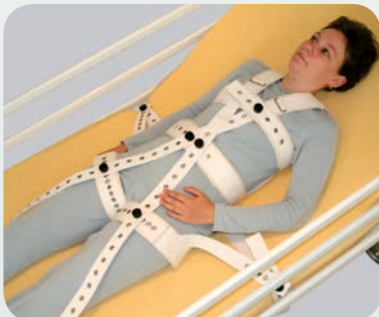


SEGUFIX®-Compact with Magnetic Lock

- patient restraint
- prevents the patient from escaping upward or downward while arms and legs remain unrestricted

Consisting of:

- 1 SEGUFIX®-Compact
- 7 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

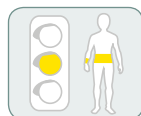

95°


Item no.	Price/€
2250/r - S	252,00
2250/r - M	260,00
2250/r - L	276,00
2250/re - S	252,00
2250/re - M	260,00
2250/re - L	276,00


SEGUFIX®-Compact:

- full mobility of arms and legs
- thigh straps prevent bruising in the crotch area
- comfortable padding
- quick release in case of emergency - only 5 locks
- compatible with other SEGUFIX®-products

Size Guide Compact

	girth	S*	M*	L*
1 torso diagonally		134 - 154 cm 52.8 - 60.6 inches	164 - 174 cm 64.6 - 68.5 inches	194 - 204 cm 76.4 - 80.3 inches
2 chest		48 - 88 cm 18.9 - 34.6 inches	75 - 115 cm 29.5 - 45.3 inches	103 - 143 cm 40.6 - 56.3 inches
3 waist		52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches
4 thigh		38 - 52 cm 15.0 - 20.5 inch	50 - 72 cm 19.7 - 28.4 inches	58 - 88 cm 22.8 - 34.7 inch

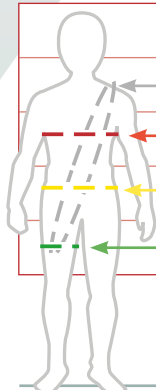

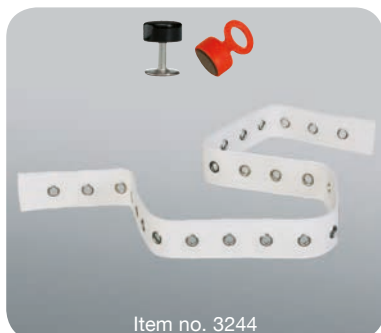

Note:

Do not use SEGUFIX®-Extension Belts to alter the size of products.

**SEGUFIX®-Patent
Locks and Magnetic
Keys not included**



SEGUFIX®-Extension Belt for Magnetic Lock

Item no. 3240 and 3240/r: with pin pocket
12 eyelets, 2 metal loops

Item no. 3241 and 3241/r: without pin pocket
12 eyelets, 2 metal loops

Item no. 3242 and 3242/r: with pin pocket
12 eyelets, 1 metal loop

Item no. 3243 and 3243/r: without pin pocket
12 eyelets, 1 metal loop

Item no. 3244 and 3244/r: without pin pocket
24 eyelets, without metal loop

Consisting of:

- 1 SEGUFIX®-Extension Belt
- 1 SEGUFIX®-Lock
- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
3240	17,70
3240/r	20,00
3241	15,40
3241/r	17,70
3242	16,40
3242/r	18,90
3243	14,10
3243/r	16,40
3244	16,10
3244/r	17,80

	3240/3140	3241/3141	3242/3142	3243/3143	3244/3144
total length*	65 cm 25.6 inches	65 cm 25.6 inches	65 cm 25.6 inches	65 cm 25.6 inches	120 cm 47.2 inches
metal loops	2	2	1	1	0
12 eyelets	X	X	X	X	
24 eyelets					X
pin pocket	X		X		

SEGUFIX®-Extension Belt for Magnetic Lock

Item no. 3140 and 3140/r: with pin pocket
12 eyelets, 2 metal loops

Item no. 3141 and 3141/r: without pin pocket
12 eyelets, 2 metal loops

Item no. 3142 and 3142/r: with pin pocket
12 eyelets, 1 metal loop

Item no. 3143 and 3143/r: without pin pocket
12 eyelets, 1 metal loop

Item no. 3144 and 3144/r: without pin pocket
24 eyelets, without metal loop

Consisting of:

- 1 SEGUFIX®-Extension Belt
- SEGUFIX®-Lock not included
- SEGUFIX®-Magnetic Key not included



Item no.	Price/€
3140	12,55
3140/r	14,85
3141	10,25
3141/r	12,55
3142	11,25
3142/r	13,75
3143	8,95
3143/r	11,25
3144	11,00
3144/r	12,70

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Extension Belt for products with Snap Lock

extends attaching straps of SEGUFIX®-Chair Belts with Snap Lock to fit wider backrests

Suitable for the following SEGUFIX®-products:

- Item no. 7240/v: SEGUFIX®-Chair Belt with Velcro® fastener
- Item no. 7240: SEGUFIX®-Chair Belt
- Item no. 7250: SEGUFIX®-Chair Belt with Crotch Strap
- Item no. 7260: SEGUFIX®-Chair Belt with Thigh Straps


Total length: 80 cm* / 31.5 inches*

Consisting of:

- 1 SEGUFIX®-Extension Belt



Item no.	Price/€
3245	11,00

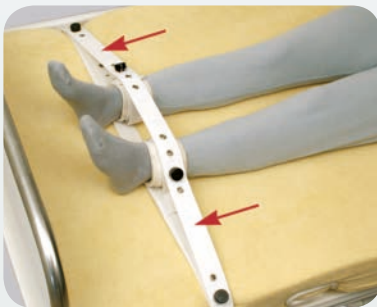

SEGUFIX®-Brace Strap for legs with eyelets for Magnetic Lock


further restricts movement in combination with the

- SEGUFIX®-Foot Restraint
- SEGUFIX®-Quick Restraint Foot
- SEGUFIX®-Thigh Restraint

Total length: 124 cm* / 48.8 inches*

Consisting of:

- 1 SEGUFIX®-Brace Strap
- SEGUFIX®-Lock not included
- SEGUFIX®-Magnetic Key not included

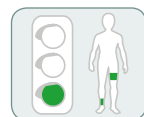

Item no.	Price/€
3110/r	14,20

Note:

To further restrict movement, the SEGUFIX®-Brace Strap for legs (item no. 3110/r) may be used in combination with the SEGUFIX®-Foot Restraint (item no. 2205, page 21), the SEGUFIX®-Quick Restraint Foot (item no. 4215/r and 4235/r, page 26-27) or the SEGUFIX®-Thigh Restraint (item no. 2211, see page 22).

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Quick Restraint Shoulder

size*	chest girth	
S	43 - 74 cm	16.9 - 29.1 inches
M	74 - 114 cm	29.1 - 44.9 inches
L	92 - 132 cm	36.2 - 52.0 inches
XL	136 - 178 cm	53.5 - 70.0 inches


SEGUFIX®-Quick Restraint Shoulder with Magnetic Lock

quick restraint in emergency situations

This item may only be used in connection with the SEGUFIX®-Quick Restraint Hand (item no. 4214, 4234) and the SEGUFIX®-Quick Restraint Foot (item no. 4215, 4235).

Consisting of:

- 1 SEGUFIX®-Quick Strap Shoulder
- 5 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

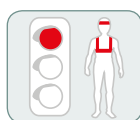

Item no.	Price/€
4213/r - S	116,50
4213/r - M	116,50
4213/r - L	122,60
4213/r - XL	140,60
4213/re - S	116,50
4213/re - M	116,50
4213/re - L	122,60
4213/re - XL	140,60



SEGUFIX®-Quick Restraint Hand


size*	wrist girth	
S	12 - 19 cm	4.7 - 7.5 inches
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Quick Restraint Hand with Magnetic Lock

quick restraint in emergency situations

Consisting of:

- 1 SEGUFIX®-Quick Strap Hand
- 4 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

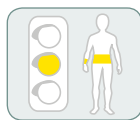

Item no.	Price/€
4214/r - S	79,10
4214/r - M	79,10
4214/r - L	94,90
4214/re - S	79,10
4214/re - M	79,10
4214/re - L	94,90



SEGUFIX®-Quick Restraint Foot


size*	ankle girth	
S	17 - 25 cm	6.7 - 9.8 inches
M	22 - 30 cm	8.7 - 11.8 inches
L	27 - 33 cm	10.6 - 13.0 inches

SEGUFIX®-Quick Restraint Foot with Magnetic Lock

quick restraint in emergency situations

Consisting of:

- 1 SEGUFIX®-Quick Strap Foot
- 4 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

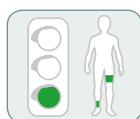

Item no.	Price/€
4215/r - S	79,10
4215/r - M	79,10
4215/r - L	94,90
4215/re - S	79,10
4215/re - M	79,10
4215/re - L	94,90

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Quick Restraint Shoulder


size*	chest girth	
S	43 - 74 cm	16.9 - 29.1 inches
M	74 - 114 cm	29.1 - 44.9 inches
L	92 - 132 cm	36.2 - 52.0 inches
XL	136 - 178 cm	53.5 - 70.0 inches

SEGUFIX®-Quick Restraint Shoulder with Magnetic Twist Lock

quick restraint in emergency situations

This item may only be used in connection with the SEGUFIX®-Quick Restraint Hand (item no. 4234, 4214) and the SEGUFIX®-Quick Restraint Foot (item no. 4235, 4215).

Consisting of:
1 SEGUFIX®-Quick Strap Shoulder
5 SEGUFIX®-Twist Locks
SEGUFIX®-Magnetic Twist Key not included

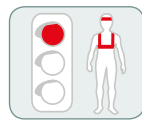

Item no.	Price/€
4233/r - S	142,50
4233/r - M	142,50
4233/r - L	148,60
4233/r - XL	166,60
4233/re - S	142,50
4233/re - M	142,50
4233/re - L	148,60
4233/re - XL	166,60




SEGUFIX®-Quick Restraint Hand

size*	wrist girth	
S	12 - 19 cm	4.7 - 7.5 inches
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches

SEGUFIX®-Quick Restraint Hand with Magnetic Twist Lock

quick restraint in emergency situations

Consisting of:
1 SEGUFIX®-Quick Strap Hand
4 SEGUFIX®-Twist Locks
1 SEGUFIX®-Magnetic Twist Key

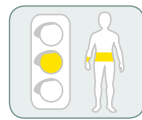

Item no.	Price/€
4234/r - S	102,30
4234/r - M	102,30
4234/r - L	118,10
4234/re - S	102,30
4234/re - M	102,30
4234/re - L	118,10




SEGUFIX®-Quick Restraint Foot

size*	ankle girth	
S	17 - 25 cm	6.7 - 9.8 inches
M	22 - 30 cm	8.7 - 11.8 inches
L	27 - 33 cm	10.6 - 13.0 inches

SEGUFIX®-Quick Restraint Foot with Magnetic Twist Lock

quick restraint in emergency situations

Consisting of:
1 SEGUFIX®-Quick Strap Foot
4 SEGUFIX®-Twist Locks
1 SEGUFIX®-Magnetic Twist Key

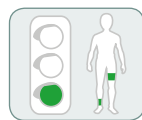

Item no.	Price/€
4235/r - S	102,30
4235/r - M	102,30
4235/r - L	118,10
4235/re - S	102,30
4235/re - M	102,30
4235/re - L	118,10



Optional for the SEGUFIX®-Quick Restraint:

- SEGUFIX®-Shoulder Additional Belt (page 19)
- SEGUFIX®-Thigh Restraint (page 22)
- SEGUFIX®-Brace Strap for legs (page 25)

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Stretcher Restraint Shoulder

size*	chest girth	
S	43 - 74 cm	16.9 - 29.1 inches
M	74 - 114 cm	29.1 - 45.9 inches
L	92 - 132 cm	36.2 - 52.0 inches

SEGUFIX®-Stretcher Restraint Shoulder with Magnetic Lock/Buckle Lock

- to secure patients on stretchers
- also suitable for ambulance services


This item may only be used in connection with the SEGUFIX®-Hand Restraint lengthened (item no. 2224) and the SEGUFIX®-Stretcher Restraint Foot (item no. 5245).

Width of stretcher: 40 cm bis max. 80 cm
(15.8 inches to max. 31.5 inches)

Attaches to the stretcher with metal buckles

Consisting of:

- 1 SEGUFIX®-Stretcher Strap Shoulder
- 3 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

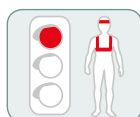

Item no.	Price/€
5243 - S	95,00
5243 - M	95,00
5243 - L	100,20



SEGUFIX®-Hand Restraint lengthened with Velcro®

size*	wrist girth	
M	16 - 24 cm	6.3 - 9.5 inches

SEGUFIX®-Hand Restraint lengthened with Velcro® fastener


- not a restraint**
- also suitable for ambulance services

Width of stretcher: 40 cm bis max. 80 cm
(15.8 inches to max. 31.5 inches)

Attaches to the stretcher with metal buckles

Consisting of:

- 2 SEGUFIX®-Hand Straps, lengthened

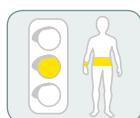

Item no.	Price/€
2224/v - M	39,50



SEGUFIX®-Stretcher Restraint Foot

size*	ankle girth	
M	22 - 30 cm	8.7 - 11.8 inches

SEGUFIX®-Stretcher Restraint Foot with Magnetic Lock/Buckle Lock


- to secure patients on stretchers
- also suitable for ambulance services

Width of stretcher: 40 cm bis max. 80 cm
(15.8 inches to max. 31.5 inches)

Attaches to the stretcher with metal buckles

Consisting of:

- 1 SEGUFIX®-Stretcher Strap Foot
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included

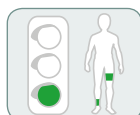

Item no.	Price/€
5245 - M	78,20



SEGUFIX®-Stretcher Restraint Foot with Velcro® fastener

size*	ankle girth	
M	22 - 30 cm	8.7 - 11.8 inches

SEGUFIX®-Stretcher Restraint Foot with Velcro® fastener


- not a restraint**
- also suitable for ambulance services

Width of stretcher: 40 cm bis max. 80 cm
(15.8 inches to max. 31.5 inches)

Attaches to the stretcher with metal buckles

Consisting of:

- 1 SEGUFIX®-Stretcher Strap Foot

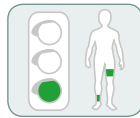

95°

Item no.	Price/€
5245/v - M	65,30



SEGUFIX®-Stretcher Restraint Complete:

- SEGUFIX®-Stretcher Restraint Shoulder, item no. 5243
- SEGUFIX®-Stretcher Restraint Foot, item no. 5245/v - M
- SEGUFIX®-Hand Restraint lengthened, item no. 2224/v - M

**Note Velcro® fastener:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in € r = reinforced e = extended bed belts
sales tax not included f = fleece v = Velcro® fastener



SEGUFIX®-Restraining Aid

length of waist belt*	approximate measurements
total length	205 cm / 80.7 inches
wide strap	59 cm / 23.2 inches
narrow strap (each side)	73 cm / 28.7 inches



SEGUFIX®-Restraining Aid with Buckle lock

length of waist belt*	approximate measurements
total length	245 cm / 96.5 inches
wide strap	59 cm / 23.2 inches
narrow strap (each side)	93 cm / 36.6 inches


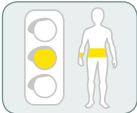
Item no.	Price/€
5218	35,80

SEGUFIX®-Restraining Aid with Magnetic Lock

- provides assistance with the restraint of patients on stretchers
- additional stabilization of the patient
- can also be fastened above the knees
- also suitable for ambulance services

Consisting of:

- 1 SEGUFIX®-Retraining Aid
- 2 SEGUFIX®-Locks
- SEGUFIX®-Magnetic Key not included




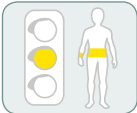
Item no.	Price/€
5218/v	35,80

SEGUFIX®-Restraining Aid with Buckle lock

- provides assistance with the restraint of patients on stretchers
- additional stabilization of the patient
- can also be fastened above the knees
- also suitable for ambulance services

Consisting of:

- 1 SEGUFIX®-Restraining Aid





*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Tightly attached
hand straps to the restraint belt
(shown in front of the stomach)



Loosely attached
hand straps to the restraint belt
(shown in front of the stomach)


SEGUFIX®-Hand Waist Restraint with Magnetic Lock

to restrain hands of patients during transfer

Use of the Hand Waist Restraint requires continuous monitoring.

Consisting of:

- 1 SEGUFIX®-Restraint Belt
- 2 SEGUFIX®-Hand Straps
- 3 SEGUFIX®-Locks
- 2 SEGUFIX®-Magnetic Keys

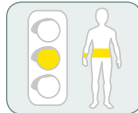

95°

Item no.	Price/€
5204/r - XS	166,10
5204/r - M	176,80
5204/r - L	195,00
5204/r - XL	253,40



Tightly attached
(shown behind the back)



Loosely attached
(shown behind the back)

SEGUFIX®-Hand Straps

size*	wrist girth	
XS	12 - 19 cm	4.7 - 7.5 inches
M	16 - 24 cm	6.3 - 9.5 inches
L	22 - 29 cm	8.7 - 11.4 inches
XL	26 - 36 cm	10.2 - 14.2 inches

SEGUFIX®-Hand Waist Restraint, waist girth

XS*	M*	L*	XL*
slim adults and children	adults	stout adults	oversize
52 - 82 cm 20.5 - 32.3 inches	80 - 120 cm 31.5 - 47.2 inches	116 - 156 cm 45.7 - 61.4 inches	140 - 180 cm 55.1 - 70.9 inches



Item no. 5204/r



Item no. 5204/r,
6230/v and 6216/v

Application:

Prevents patients from injuring themselves or others, e. g. accompanying personnel.

Attention:

The SEGUFIX®-Transport System does not replace the vehicles own seat belt system. The SEGUFIX®-Transport Belts may only be used as additional support or for positioning of the patient.

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Transport Waist Belt with Velcro® fastener

size*	waist girth	
S	52 - 82 cm	20.5 - 32.3 inches
M	80 - 120 cm	31.5 - 47.2 inches
L	116 - 156 cm	45.7 - 61.4 inches

SEGUFIX®-Transport Waist Belt with Velcro® fastener

to support patients in wheelchairs, automobiles and other suitable seats

Application:

SEGUFIX®-Backrest Strap required

(item no. 6230/v)

SEGUFIX®-Suspender Belt if needed

(item no. 6220)

The Velcro® fastener must overlap at least

25 cm / 9.8 inches

Width of waist belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Transport Waist Belt



Item no.	Price/€
6210/v - S	34,80
6210/v - M	37,80
6210/v - L	40,90



SEGUFIX®-Transport Waist Belt

size*	waist girth	
S	52 - 82 cm	20.5 - 32.3 inches
M	80 - 120 cm	31.5 - 47.2 inches
L	116 - 156 cm	45.7 - 61.4 inches

SEGUFIX®-Transport Waist Belt with Velcro® fastener/Magnetic Lock

to support patients in wheelchairs, automobiles and other suitable seats

Application:

SEGUFIX®-Backrest Strap required

(item no. 6230/v)

SEGUFIX®-Suspender Belt if needed

(item no. 6220)

Width of waist belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Transport Waist Belt

- 1 SEGUFIX®-Lock

- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
6216/v - S	45,60
6216/v - M	48,70
6216/v - L	51,80



SEGUFIX®-Backrest Strap Velcro®

size*	circumference of backrest vertical	
S	78 - 103 cm	30.7 - 40.6 inches
M	98 - 123 cm	38.6 - 48.4 inches
L	118 - 143 cm	46.5 - 56.3 inches
XL	142 - 168 cm	55.9 - 66.1 inches

SEGUFIX®-Backrest Strap with Velcro® fastener

To attach the SEGUFIX®-Transport Waist Belt (item no. 6210/v or 6216/v or 62116/v) and the SEGUFIX®-Suspender Belt (item no. 6220) to the backrest.

May not be used on folding wheelchairs with a soft backrest. The Backrest Strap is only suitable for chairs and wheelchairs with **a solid backrest and a gap** between backrest and seat.

The Velcro® fastener must overlap at least

25 cm / 9.8 inches

Width of waist belt: 18 cm* / 7.1 inches*

Consisting of:

- 1 SEGUFIX®-Backrest Strap



Item no.	Price/€
6230/v - S	34,80
6230/v - M	37,80
6230/v - L	40,90
6230/v - XL	44,00



Item no. 6116/v, 6230/v



Item no. 6210/v, 6230/v

Application: To support patients in wheelchairs, automobiles and other suitable chairs.

Attention: The SEGUFIX®-Transport System does not replace the vehicles own seat belt system. The SEGUFIX®-Transport System may only be used as additional support or for positioning of the patient.



Note: Use size S for patients up to a body height of 140 cm/4 feet 8 inches. For taller patients use size M.

SEGUFIX®-Suspenders Belt with Snap Lock

- to support patients in wheelchairs, automobiles and other suitable seats
- holds the patient in an upright position, prevents patient from sliding down

Application:
SEGUFIX®-Backrest Strap (item no. 6230/v) and SEGUFIX®-Transport Waist Belt (item no. 6210/v or 6216/v) required

Consisting of:
1 SEGUFIX®-Suspenders Belt



Item no.	Price/€
6220 - S	63,90
6220 - M	70,00



view from rear

SEGUFIX®-Support Belt with Magnetic Lock

- supports the sitting patient
- supports the upper body while preventing the patient from sliding off of the seat

Attaches behind the backrest with Lock:


Item no. 6240 consisting of:
1 SEGUFIX®-Support Belt
1 SEGUFIX®-Lock
1 SEGUFIX®-Magnetic Key

Item no. 6244 consisting of:
1 SEGUFIX®-Support Belt
1 SEGUFIX®-Crotch Pad
1 SEGUFIX®-Lock
1 SEGUFIX®-Magnetic Key



Item no.	Price/€
6240 - S	128,90
6240 - M	135,90
6240 - L	139,80
6244 - S	158,40
6244 - M	165,40
6244 - L	169,30

Size guide Support Belt

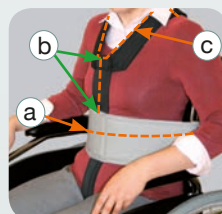

size*	a waist belt		b connection		c shoulder collar	
S	90 - 130 cm	35.4 - 51.2 inches	10 cm	3.9 inches	80 cm	31.5 inches
M	90 - 130 cm	35.4 - 51.2 inches	15 cm	5.9 inches	80 cm	31.5 inches
L	122 - 162 cm	48.0 - 63.8 inches	15 cm	5.9 inches	95 cm	37.4 inches

- a) Girth of patient and backrest at waist height.
b) Connecting strap between waist belt and shoulder collar.
c) Inner circumference of the shoulder collar.

Measurements are taken while the patient is sitting.



SEGUFIX®-Crotch Pad for the Support Belt

padding for the crotch area

Application:
Only compatible with the SEGUFIX®-Support Belt (Item no. 6240 and 6244).

Consisting of:
1 SEGUFIX®-Crotch Pad



Item no.	Price/€
6040	29,50

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

Locks & Keys
Item no. 1106 - 1215

Bed Restraint System
Item no. 2203 - 2250

Auxiliary Belts
Item no. 3110 - 3245

Quick Restraint
Item no. 4213 - 4235

Mobile Restraint
Item no. 5204 - 5245

Transport System
Item no. 6040 - 6276

Chair Belt System
Item no. 7240 - 7280

More Products
Item no. 8010 - 8022



SEGUFIX®-Multi Belt with Velcro® fastener

- support belt for patients in chairs, armchairs and other suitable seats
- also suitable for ambulance stretchers, or for support on shower chairs

The Velcro® fastener must overlap at least 25 cm / 9.8 inches

Width of waist belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Multi Belt



Item no.	Price/€
6250/v - XS	41,70
6250/v - S	41,70
6250/v - M	49,90
6250/v - L	49,90



SEGUFIX®-Multi Belt with Velcro® fastener and Magnetic Lock

- support belt for patients in chairs, armchairs and other suitable seats
- also suitable for ambulance stretchers, or for support on shower chairs (without SEGUFIX®-Lock)

Width of waist belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Multi Belt
- 1 SEGUFIX®-Lock
- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
6256/v - XS	53,20
6256/v - S	53,20
6256/v - M	61,40
6256/v - L	61,40



SEGUFIX®-Multi Belt

size*	total length		circumference** with Velcro® fastener (item no. 6250/v)		circumference** with Magnetic Lock (item no. 6256/v)	
XS	160 cm	63.0 inches	100 - 135 cm	39.4 - 53.2 inches	110 - 150 cm	43.3 - 59.1 inches
S	180 cm	70.9 inches	120 - 155 cm	49.2 - 61.0 inches	130 - 170 cm	51.2 - 67.0 inches
M	200 cm	78.7 inches	135 - 175 cm	53.2 - 68.9 inches	150 - 190 cm	59.1 - 74.8 inches
L	240 cm	94.5 inches	175 - 215 cm	68.9 - 84.7 inches	190 - 230 cm	74.8 - 90.6 inches

**total girth (patient and backrest/stretcher)



SEGUFIX®-Protective Cover

Item no. 8013 / € 9,90

see page 41!

(fig. item no. 8013 and 6250/v)

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Pelvic Belt with Snap Lock

support belt for suitable chairs

Length of grey Pelvic Belt: 74 cm* / 29.1 inches*
Width of Pelvic Belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Pelvic Belt



Item no.	Price/€
6260	43,50



SEGUFIX®-Pelvic Belt with Magnetic Lock

support belt for suitable chairs


Length of grey Pelvic Belt: 74 cm* / 29.1 inches*
Width of Pelvic Belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Pelvic Belt
- 2 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
6266	60,30

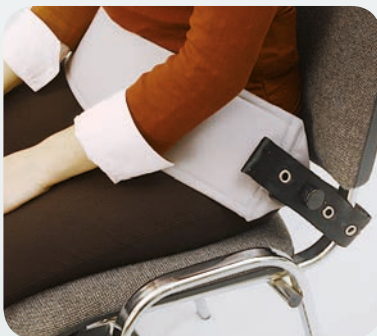

SEGUFIX®-Pelvic Belt with Crotch Strap and Magnetic Lock

- supports patients in chairs
- prevents patients from sliding off of the chair

Length of grey Pelvic Belt: 74 cm* / 29.1 inches*
Width of Pelvic Belt: 14 cm* / 5.5 inches*

Consisting of:

- 1 SEGUFIX®-Pelvic Belt with Crotch Strap
- 1 SEGUFIX®-Lock
- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
6276	68,70



The Anti-Slide-Mat

prevents the patient from sliding off of the seat. Allows for easy sliding in one direction, sliding in the other direction is prevented.

Washable in hot wash cycles up to 80°C.

item no. 8017 / € 66,15

The SEGUFIX®-Protective Cover

protects SEGUFIX®-Transport belts from staining. Washable in hot wash cycles up to 95°C, autoclavable and bacteria resistant.

item no. 8013

€ 9,90



SEGUFIX®-Chair Belt System



Capacity of the snap lock up to 70 kg



SEGUFIX®-Chair Belt with Velcro® fastener

to support and position patients in wheelchairs and other suitable seats

Circumference of backrest:
70 - 150 cm* / 27.6 - 59.1 inches*
Width of waist belt: 14 cm* / 5.5 inches*
The Velcro® fastener must overlap at least 25 cm / 9.8 inches

Attaches behind the backrest with Snap Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt

(Extension Belt with Snap Lock, item no. 3245, see Auxiliary Belts page 25)



Item no.	Price/€
7240/v - S	74,70
7240/v - M	80,90
7240/v - L	91,60



All Chair Belts are also available with fleece padding!



SEGUFIX®-Chair Belt with Magnetic Lock

to support and position patients in wheelchairs and other suitable seats

Circumference of backrest:
70 - 150 cm* / 27.6 - 59.1 inches*
Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Snap Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt
- 1 SEGUFIX®-Lock
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Snap Lock, item no. 3245, see Auxiliary Belts page 25)



Item no.	Price/€
7240 - S	77,70
7240 - M	83,90
7240 - L	94,60
7240/f - S	112,00
7240/f - M	118,80
7240/f - L	138,30



SEGUFIX®-Chair Belt with Magnetic Lock

to support and position patients in wheelchairs and other suitable seats

Circumference of backrest:
70 - 115 cm* / 27.6 - 45.3 inches*
Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt
- 2 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Magnetic Lock, item no. 3244, see Auxiliary Belts page 24)



Item no.	Price/€
7246 - S	95,80
7246 - M	102,90
7246 - L	115,20
7246/f - S	130,10
7246/f - M	137,80
7246/f - L	158,90

SEGUFIX®-Chair Belt

size*	waist girth	
S	52 - 82 cm	20.5 - 32.3 inches
M	80 - 120 cm	31.5 - 47.2 inches
L	116 - 156 cm	45.7 - 61.4 inches

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Capacity of the snap lock up to 70 kg

SEGUFIX®-Chair Belt with Crotch Strap and Magnetic Lock

- to support and position patients in wheelchairs and other suitable seats
- prevents the patient from sliding off of the seat

Circumference of backrest:
70 - 150 cm* / 27.6 - 59.1 inches*
Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Snap Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt with Crotch Strap
- 1 SEGUFIX®-Lock
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Snap Lock, item no. 3245, see Auxiliary Belts page 25)



Item no.	Price/€
7250 - S	112,30
7250 - M	118,50
7250 - L	129,20
7250/f - S	146,60
7250/f - M	153,40
7250/f - L	172,90



SEGUFIX®-Chair Belt with Crotch Strap and Magnetic Lock

- to support and position patients in wheelchairs and other suitable seats
- prevents the patient from sliding off of the seat

Circumference of backrest:
70 - 115 cm* / 27.6 - 45.3 inches*
Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt with Crotch Strap
- 2 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Magnetic Lock, item no. 3244, see Auxiliary Belts page 24)



Item no.	Price/€
7256 - S	130,40
7256 - M	137,50
7256 - L	149,80
7256/f - S	164,70
7256/f - M	172,40
7256/f - L	193,50

SEGUFIX®-Chair Belt with Crotch Strap

size*	waist girth
S	52 - 82 cm 20.5 - 32.3 inches
M	80 - 120 cm 31.5 - 47.2 inches
L	116 - 156 cm 45.7 - 61.4 inches



Item no. 3244

SEGUFIX®-Extension Belt with Magnetic Lock

see page 24!



Item no. 3245

SEGUFIX®-Extension Belt for products with Snap Lock

see page 25!

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Capacity of the snap lock up to 70 kg



All Chair Belts are also available with fleece padding!

SEGUFIX®-Chair Belt with Thigh Straps

size*	waist girth	
S	52 - 82 cm	20.5 - 32.3 inches
M	80 - 120 cm	31.5 - 47.2 inches
L	116 - 156 cm	45.7 - 61.4 inches

size*	thigh girth	
S	38 - 52 cm	15.0 - 20.5 inches
M	50 - 72 cm	19.7 - 28.4 inches
L	58 - 88 cm	22.8 - 34.7 inches

SEGUFIX®-Chair Belt with Thigh Straps and Magnetic Lock

- to support and position patients in wheelchairs and other suitable seats
- prevents the patient from sliding off of the seat

Circumference of backrest:

70 - 150 cm* / 27.6 - 59.1 inches*

Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Snap Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt with Thigh Straps
- 3 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Magnetic Lock, item no. 3244, see Auxiliary Belts page 25)



Item no.	Price/€
7260 - S	149,50
7260 - M	159,40
7260 - L	170,30
7260/f - S	183,80
7260/f - M	194,30
7260/f - L	214,00

SEGUFIX®-Chair Belt with Thigh Straps and Magnetic Lock

- to support and position patients in wheelchairs and other suitable seats
- prevents the patient from sliding off of the seat

Circumference of backrest:

70 - 115 cm* / 27.6 - 45.3 inches*

Width of waist belt: 14 cm* / 5.5 inches*

Attaches behind the backrest with Lock

Consisting of:

- 1 SEGUFIX®-Chair Belt with Thigh Straps
- 4 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key

(Extension Belt with Magnetic Lock, item no. 3244, see Auxiliary Belts page 24)



Item no.	Price/€
7266 - S	166,90
7266 - M	176,80
7266 - L	187,70
7266/f - S	201,20
7266/f - M	211,70
7266/f - L	231,40



*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



Transport Hand Strap Velcro®

wrist girth	
16 - 24 cm*	6.3 - 9.5 inches*

SEGUFIX®-Hand Strap Transport with Velcro® fastener

- to position the hands of patients on wheelchairs and other chairs with suitable armrests
- not a restraint**

Consisting of:

- 1 SEGUFIX®-Hand Strap Transport (for one hand)



Item no.	Price/€
7270/v	14,80



Transport Foot Restraint Velcro®

ankle girth	
22 - 30 cm*	8.7 - 11.8 inches*

SEGUFIX®-Foot Strap Transport with Velcro® fastener

- to position the feet of patients in wheelchairs and other suitable chairs
- not a restraint**

Attachment to a wheel chair with metal buckles

Girth: 80 - 100 cm* / 31.5 - 39.4 inches*

Consisting of:

- 1 SEGUFIX®-Foot Strap Transport (for both feet)



Item no.	Price/€
7280/v	38,30



Transport Foot Restraint

ankle girth	
22 - 30 cm*	8.7 - 11.8 inches*

SEGUFIX®-Foot Strap Transport with Magnetic Lock

to position the feet of patients in wheelchairs and other suitable chairs

Attachment to a wheel chair with Magnetic Lock

Girth: 68 - 112 cm* / 26.8 - 44.1 inches*

Consisting of:

- 1 SEGUFIX®-Foot Strap Transport (for both feet)
- 3 SEGUFIX®-Locks
- 1 SEGUFIX®-Magnetic Key



Item no.	Price/€
7280	38,30

**Note Velcro® fastener:

However some jurisdictions might consider Velcro® products a restraint, should the patient not be able to remove them independently. Please check your applicable legal requirements.

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Sheepskin Padding for SEGUFIX®-Standard with Crotch Strap or Standard with Thigh Straps Item no. 2221 and item no. 2121 Item no. 2231 and item no. 2131

- skin-friendly
- compensates for changing temperature
- quick absorption and dispersion of perspiration



Item no.	Price/€
8012 - XS/S	42,90
8012 - M	57,90
8012 - L	61,90

(see Waist Size Guide page 10)



SEGUFIX®-Sheepskin Wheelchair Cover with Hood

- skin-friendly
- compensates for changing temperature
- quick absorption and dispersion of perspiration
- size: approx. 40 x 100 cm* / 15.8 x 39.4 inches*



Item no.	Price/€
8011	91,60



SEGUFIX®-Sheepskin Bed Cover

- skin-friendly
- compensates for changing temperature
- quick absorption and dispersion of perspiration
- size: approx. 90 x 180 cm* / 35.4 x 70.9 inches*



Item no.	Price/€
8010	256,60

Interesting facts about medical sheepskins:

Medical sheepskins are natural products which help to ensure a healthy and constant temperature. This gives a relaxing and revitalizing effect over the entire applied area.

Doctors recommend natural sheepskin especially for use with bedsores, allergies, neuralgia, nervousness, poor circulation, rheumatism, sciatica, back pain and limb pain.

Other positive qualities of natural sheepskin:

- Immediate absorption of perspiration and seven times faster dispersion into air than synthetic materials. Body toxins secreted with perspiration are broken down.
- Sheepskins are a natural product and therefore have a self-cleaning effect.
- Natural sheepskin is at body temperature and never feels cold. Natural breathing of the patient's skin is ensured.
- Temperature balanced, suitable for summer and winter use.
- Non-static.

Instructions for care:

Washing machine:

- Washable at warm wash cycles up to 30 - 40°C, no spin-cycle.
- Use only wool or delicate detergents.

Hand washing:

- Wash quickly in lukewarm water (30 - 40°C).
- Rinse thoroughly with lukewarm water and gently wring-out water.
- Use only wool or delicate detergents.

How to dry the sheepskin:

- Hang dry or tumble dry on delicate cycle at low temperature.
Caution: Avoid direct sunlight!
- Before the sheepskin is completely dry, pull to reshape it. If needed, brush with a sturdy brush.

Note:

Do not leave sheepskin submerged for an extended period of time (do not soak).

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Pull-up Line

helps the patient to sit up in an ergonomic manner, preventing damage to the spine

Total length: 160 cm* / 63.0 inches*

Consisting of:

- 1 SEGUFIX®-Pull-up Line



Item no.	Price/€
8016	21,00



SEGUFIX®-Protective Cover

The SEGUFIX®-Protective Cover protects all SEGUFIX®-Transport-Belts listed below from staining:

- SEGUFIX®-Multi Belt, item no. 6250/v
- SEGUFIX®-Multi Belt, item no. 6256/v
- SEGUFIX®-Multi Belt, item no. 6156/v

(the SEGUFIX®-Lock cannot be used with the Protective Cover)

- SEGUFIX®-Pelvic Belt, item no. 6260
- SEGUFIX®-Pelvic Belt, item no. 6266
- SEGUFIX®-Pelvic Belt, item no. 6166

The SEGUFIX®-Protective Cover is washable in hot wash cycles up to 95°C and autoclavable.

The Protective Cover is bacteria resistant.

Material: 100% Polyester coated with 100% Polyurethane

Consisting of:

- 1 SEGUFIX®-Protective Cover



Item no.	Price/€
8013	9,90



SEGUFIX®-Pelvic Belt (item no. 6266) with Protective Cover (item no. 8013)



Anti-Slide-Mat

- prevents weak patients from sliding off of their chair
- great for use on chairs, care chairs, wheelchairs, in bed, office chairs etc.
- allows for easy sliding in one direction, sliding in the other direction is prevented
- not for restless patients
- the Anti-Slide-Mat does not work when wet.

Maximum tensile load: 250 kg

Size: 37 x 43 cm* / 14.6 x 17.0 inches *

Consisting of:

- 1 Anti-Slide-Mat



Item no.	Price/€
8017	66,15

Note:

- waschable up to 80°C
- no detergent, no bleach
- Dryer instructions: (beginning temperature 80°C, ending temperature 60°C)
- may not be disinfected
- after washing the Anti-Slide-Mat must be completely dry before use to ensure proper function

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener



SEGUFIX®-Transport Bag

- to store several SEGUFIX® products on various hospital wards
- to protect SEGUFIX® products
- separate pockets on each end
- handles and detachable carrying strap
- useful for transporting SEGUFIX® products, also great for in-house training
- size: approx. 62 cm x 32 cm x 28 cm / 24.4 inches x 13.0 inches x 10.6 inches
- colour: black (white SEGUFIX® print)

Consisting of:

- 1 SEGUFIX®-Transport Bag

Item no.	Price/€
8014	29,80




SEGUFIX®-Response Bag

- quick access for fast response
- protects SEGUFIX® products
- well arranged storage for your SEGUFIX®-sets, e.g. 1 SEGUFIX®-Complett, 1 SEGUFIX®-Quick Restraint etc.
- space saver, several bags may be stacked for organized storage
- pockets for the SEGUFIX®-Manual and additional SEGUFIX®-Locks and Magnetic Keys etc.
- handle and detachable carrying strap
- bag is padded
- size: approx. 42 cm x 26 cm x 10 cm / 16.5 inches x 10.2 inches x 3.9 inches
- colour: red / black, (white SEGUFIX® print)

Consisting of:

- 1 SEGUFIX®-Response Bag (contents not included)

Item no.	Price/€
8015	28,60

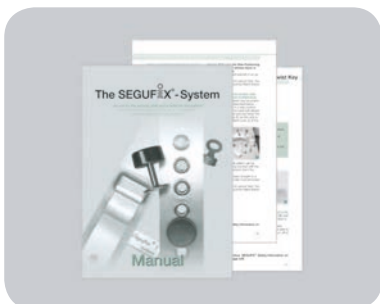

SEGUFIX®-Manual

The SEGUFIX®-Manual includes the individual instructions of all SEGUFIX®-products with illustrated step-by-step directions.

- Size: DIN A5
- Black and white photos and illustrations

Consisting of:

- 1 SEGUFIX®-Manual

Item no.	Price/€
8019/GB	3,50



SEGUFIX®-Laundry Net

protects strap material during wash cycle

Size: 60 x 40 cm* / 23.6 x 15.8 inches*

Consisting of:

- 1 SEGUFIX®-Laundry Net with integrated drawstring



Item no.	Price/€
8022	5,98




*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included


r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener


SEGUFIX®-Locks and Keys (see page 12 - 13)

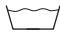Item no.	Product name	Price/€	
1106	SEGUFIX®-Button, black	4,60	—
1117	SEGUFIX®-Metal Pin	1,90	—
1110	SEGUFIX®-Twist Button, green	6,40	—
1111	SEGUFIX®-Square Pin	5,20	—
1113	Push Button	17,90	—
1114	Push Pin	1,90	—

SEGUFIX®-Bed Restraint System (see page 14 - 23)

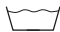Item no.	Product name	Price/€	
2005 - XS	SEGUFIX®-Connecting Belt	14,00	95°C
2005 - M	SEGUFIX®-Connecting Belt	15,10	95°C
2005 - L	SEGUFIX®-Connecting Belt	15,10	95°C
2005/e - XS	SEGUFIX®-Connecting Belt, extended	14,00	95°C
2005/e - M	SEGUFIX®-Connecting Belt, extended	15,10	95°C
2005/e - L	SEGUFIX®-Connecting Belt, extended	15,10	95°C
2005/r - XS	SEGUFIX®-Connecting Belt, reinforced	16,10	95°C
2005/r - M	SEGUFIX®-Connecting Belt, reinforced	16,80	95°C
2005/r - L	SEGUFIX®-Connecting Belt, reinforced	16,80	95°C
2005/re - XS	SEGUFIX®-Connecting Belt, reinforced, extended	16,10	95°C
2005/re - M	SEGUFIX®-Connecting Belt, reinforced, extended	16,80	95°C
2005/re - L	SEGUFIX®-Connecting Belt, reinforced, extended	16,80	95°C
2005/v - XS	SEGUFIX®-Connecting Belt, Velcro®	14,00	95°C
2005/v - M	SEGUFIX®-Connecting Belt, Velcro®	15,10	95°C
2005/ev - XS	SEGUFIX®-Connecting Belt, Velcro®, extended	14,00	95°C
2005/ev - M	SEGUFIX®-Connecting Belt, Velcro®, extended	15,10	95°C
2103 - XS	SEGUFIX®-Shoulder Strap	30,60	95°C
2103 - S	SEGUFIX®-Shoulder Strap	34,00	95°C
2103 - M	SEGUFIX®-Shoulder Strap	34,00	95°C
2103 - L	SEGUFIX®-Shoulder Strap	37,10	95°C
2103/r - XS	SEGUFIX®-Shoulder Strap, reinforced	35,20	95°C
2103/r - S	SEGUFIX®-Shoulder Strap, reinforced	39,10	95°C
2103/r - M	SEGUFIX®-Shoulder Strap, reinforced	39,10	95°C
2103/r - L	SEGUFIX®-Shoulder Strap, reinforced	43,40	95°C
2104 - XS	SEGUFIX®-Hand Strap	11,00	95°C
2104 - M	SEGUFIX®-Hand Strap	12,00	95°C
2104 - L	SEGUFIX®-Hand Strap	14,40	95°C
2104/r - XS	SEGUFIX®-Hand Strap, reinforced	12,70	95°C
2104/r - M	SEGUFIX®-Hand Strap, reinforced	13,80	95°C
2104/r - L	SEGUFIX®-Hand Strap, reinforced	16,50	95°C

SEGUFIX® - Bed Restraint System (see page 14 - 23)

Item no.	Product name	Price/€	
2105 - XS	SEGUFIX®-Foot Strap	15,30	95°C
2105 - M	SEGUFIX®-Foot Strap	16,60	95°C
2105 - L	SEGUFIX®-Foot Strap	22,00	95°C
2105/r - XS	SEGUFIX®-Foot Strap, reinforced	19,20	95°C
2105/r - M	SEGUFIX®-Foot Strap, reinforced	20,70	95°C
2105/r - L	SEGUFIX®-Foot Strap, reinforced	26,80	95°C
2105/v - XS	SEGUFIX®-Foot Strap, Velcro®	21,10	95°C
2105/v - M	SEGUFIX®-Foot Strap, Velcro®	22,10	95°C
2111 - S	SEGUFIX®-Thigh Belt	62,80	95°C
2111 - M	SEGUFIX®-Thigh Belt	62,80	95°C
2111 - L	SEGUFIX®-Thigh Belt	69,10	95°C
2111/e - S	SEGUFIX®-Thigh Belt, extended	62,80	95°C
2111/e - M	SEGUFIX®-Thigh Belt, extended	62,80	95°C
2111/e - L	SEGUFIX®-Thigh Belt, extended	69,10	95°C
2111/r - S	SEGUFIX®-Thigh Belt, reinforced	74,10	95°C
2111/r - M	SEGUFIX®-Thigh Belt, reinforced	74,10	95°C
2111/r - L	SEGUFIX®-Thigh Belt, reinforced	81,50	95°C
2111/re - S	SEGUFIX®-Thigh Belt, reinforced, extended	74,10	95°C
2111/re - M	SEGUFIX®-Thigh Belt, reinforced, extended	74,10	95°C
2111/re - L	SEGUFIX®-Thigh Belt, reinforced, extended	81,50	95°C
2112	SEGUFIX®-Shoulder Additional Strap	19,70	95°C
2112/r	SEGUFIX®-Shoulder Additional Strap, reinforced	24,30	95°C
2117	SEGUFIX®-Head Positioning Strap	70,60	95°C
2117/e	SEGUFIX®-Head Positioning Strap, extended	70,60	95°C
2121 - XS	SEGUFIX®-Standard with Crotch Strap	110,40	95°C
2121 - S	SEGUFIX®-Standard with Crotch Strap	116,50	95°C
2121 - M	SEGUFIX®-Standard with Crotch Strap	116,50	95°C
2121 - L	SEGUFIX®-Standard with Crotch Strap	127,20	95°C
2121/e - XS	SEGUFIX®-Standard with Crotch Strap, extended	110,40	95°C
2121/e - S	SEGUFIX®-Standard with Crotch Strap, extended	116,50	95°C
2121/e - M	SEGUFIX®-Standard with Crotch Strap, extended	116,50	95°C
2121/e - L	SEGUFIX®-Standard with Crotch Strap, extended	127,20	95°C
2121/r - XS	SEGUFIX®-Standard with Crotch Strap, reinforced	125,00	95°C
2121/r - S	SEGUFIX®-Standard with Crotch Strap, reinforced	132,00	95°C
2121/r - M	SEGUFIX®-Standard with Crotch Strap, reinforced	132,00	95°C
2121/r - L	SEGUFIX®-Standard with Crotch Strap, reinforced	144,50	95°C
2121/r - XL	SEGUFIX®-Standard with Crotch Strap, reinforced	172,80	95°C

*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included


r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

SEGUFIX®-Bed Restraint System (see page 14 - 23)

Item no.	Product name	Price/€	
2121/re - XS	SEGUFIX®-Standard with Crotch Strap, reinforced, extended	125,00	95°C
2121/re - S	SEGUFIX®-Standard with Crotch Strap, reinforced, extended	132,00	95°C
2121/re - M	SEGUFIX®-Standard with Crotch Strap, reinforced, extended	132,00	95°C
2121/re - L	SEGUFIX®-Standard with Crotch Strap, reinforced, extended	144,50	95°C
2121/re - XL	SEGUFIX®-Standard with Crotch Strap, reinforced, extended	172,80	95°C
2121/f - XS	SEGUFIX®-Standard with Crotch Strap Fleece	144,70	95°C
2121/f - S	SEGUFIX®-Standard with Crotch Strap Fleece	151,40	95°C
2121/f - M	SEGUFIX®-Standard with Crotch Strap Fleece	151,40	95°C
2121/f - L	SEGUFIX®-Standard with Crotch Strap Fleece	170,90	95°C
2121/ef - XS	SEGUFIX®-Standard with Crotch Strap Fleece, extended	144,70	95°C
2121/ef - S	SEGUFIX®-Standard with Crotch Strap Fleece, extended	151,40	95°C
2121/ef - M	SEGUFIX®-Standard with Crotch Strap Fleece, extended	151,40	95°C
2121/ef - L	SEGUFIX®-Standard with Crotch Strap Fleece, extended	170,90	95°C
2121/rf - XS	SEGUFIX®-Standard Crotch Strap Fleece, reinforced	163,95	95°C
2121/rf - S	SEGUFIX®-Standard Crotch Strap Fleece, reinforced	171,45	95°C
2121/rf - M	SEGUFIX®-Standard Crotch Strap Fleece, reinforced	171,45	95°C
2121/rf - L	SEGUFIX®-Standard Crotch Strap Fleece, reinforced	194,05	95°C
2121/rf - XL	SEGUFIX®-Standard Crotch Strap Fleece, reinforced	247,30	95°C
2121/ref - XS	SEGUFIX®-Standard with Crotch Strap Fleece, reinforced, extended	163,95	95°C
2121/ref - S	SEGUFIX®-Standard with Crotch Strap Fleece, reinforced, extended	171,45	95°C
2121/ref - M	SEGUFIX®-Standard with Crotch Strap Fleece, reinforced, extended	171,45	95°C
2121/ref - L	SEGUFIX®-Standard with Crotch Strap Fleece, reinforced, extended	194,05	95°C
2121/ref - XL	SEGUFIX®-Standard with Crotch Strap Fleece, reinforced, extended	247,30	95°C
2124 - M	SEGUFIX®-Hand Strap, extended	14,60	95°C
2124 - L	SEGUFIX®-Hand Strap, extended	19,50	95°C
2124/r - M	SEGUFIX®-Hand Strap, reinforced, extended	18,20	95°C
2124/r - L	SEGUFIX®-Hand Strap, reinforced, extended	24,20	95°C
2124/v - M	SEGUFIX®-Hand Strap, extended, Velcro®	21,60	95°C
2131 - S	SEGUFIX®-Standard with Thigh Straps	147,40	95°C
2131 - M	SEGUFIX®-Standard with Thigh Straps	147,40	95°C
2131 - L	SEGUFIX®-Standard with Thigh Straps	158,10	95°C
2131/e - S	SEGUFIX®-Standard with Thigh Straps, extended	147,40	95°C
2131/e - M	SEGUFIX®-Standard with Thigh Straps, extended	147,40	95°C
2131/e - L	SEGUFIX®-Standard with Thigh Straps, extended	158,10	95°C
2131/r - S	SEGUFIX®-Standard with Thigh Straps, reinforced	163,10	95°C
2131/r - M	SEGUFIX®-Standard with Thigh Straps, reinforced	163,10	95°C
2131/r - L	SEGUFIX®-Standard with Thigh Straps, reinforced	175,40	95°C
2131/r - XL	SEGUFIX®-Standard with Thigh Straps, reinforced	206,70	95°C

SEGUFIX® - Bed Restraint System (see page 14 - 23)

Item no.	Product name	Price/€	
2131/re - S	SEGUFIX®-Standard with Thigh Straps, reinforced, extended	163,10	95°C
2131/re - M	SEGUFIX®-Standard with Thigh Straps, reinforced, extended	163,10	95°C
2131/re - L	SEGUFIX®-Standard with Thigh Straps, reinforced, extended	175,40	95°C
2131/re - XL	SEGUFIX®-Standard with Thigh Straps, reinforced, extended	206,70	95°C
2131/f - S	SEGUFIX®-Standard with Thigh Straps Fleece	182,30	95°C
2131/f - M	SEGUFIX®-Standard with Thigh Straps Fleece	182,30	95°C
2131/f - L	SEGUFIX®-Standard with Thigh Straps Fleece	201,80	95°C
2131/ef - S	SEGUFIX®-Standard with Thigh Straps Fleece, extended	182,30	95°C
2131/ef - M	SEGUFIX®-Standard with Thigh Straps Fleece, extended	182,30	95°C
2131/ef - L	SEGUFIX®-Standard with Thigh Straps Fleece, extended	201,80	95°C
2131/rf - S	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced	202,55	95°C
2131/rf - M	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced	202,55	95°C
2131/rf - L	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced	224,95	95°C
2131/rf - XL	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced	281,20	95°C
2131/ref - S	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced, extended	202,55	95°C
2131/ref - M	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced, extended	202,55	95°C
2131/ref - L	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced, extended	224,95	95°C
2131/ref - XL	SEGUFIX®-Standard with Thigh Straps Fleece, reinforced, extended	281,20	95°C
2150/r - S	SEGUFIX®-Compact, reinforced	209,80	95°C
2150/r - M	SEGUFIX®-Compact, reinforced	217,80	95°C
2150/r - L	SEGUFIX®-Compact, reinforced	233,80	95°C
2150/re - S	SEGUFIX®-Compact, reinforced, extended	209,80	95°C
2150/re - M	SEGUFIX®-Compact, reinforced, extended	217,80	95°C
2150/re - L	SEGUFIX®-Compact, reinforced, extended	233,80	95°C


*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included


r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

SEGUFIX®-Auxiliary Belts (see page 24 - 25)

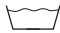Item no.	Product name	Price/€	
3140	SEGUFIX®-Extension Belt	12,55	95°C
3140/r	SEGUFIX®-Extension Belt, reinforced	14,85	95°C
3141	SEGUFIX®-Extension Belt	10,25	95°C
3141/r	SEGUFIX®-Extension Belt, reinforced	12,55	95°C
3142	SEGUFIX®-Extension Belt	11,25	95°C
3142/r	SEGUFIX®-Extension Belt, reinforced	13,75	95°C
3143	SEGUFIX®-Extension Belt	8,95	95°C
3143/r	SEGUFIX®-Extension Belt, reinforced	11,25	95°C
3144	SEGUFIX®-Extension Belt	11,00	95°C
3144/r	SEGUFIX®-Extension Belt, reinforced	12,70	95°C
3110/r	SEGUFIX®-Brace Strap for legs, reinforced	14,20	95°C

SEGUFIX®-Quick Restraint (see page 26 - 27)

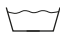Item no.	Product name	Price/€	
4113/r - S	SEGUFIX®-Quick Strap Shoulder, reinforced	90,90	95°C
4113/r - M	SEGUFIX®-Quick Strap Shoulder, reinforced	90,90	95°C
4113/r - L	SEGUFIX®-Quick Strap Shoulder, reinforced	97,00	95°C
4113/r - XL	SEGUFIX®-Quick Strap Shoulder, reinforced	115,00	95°C
4113/re - S	SEGUFIX®-Quick Strap Shoulder, reinforced, extended	90,90	95°C
4113/re - M	SEGUFIX®-Quick Strap Shoulder, reinforced, extended	90,90	95°C
4113/re - L	SEGUFIX®-Quick Strap Shoulder, reinforced, extended	97,00	95°C
4113/re - XL	SEGUFIX®-Quick Strap Shoulder, reinforced, extended	115,00	95°C
4114/r - S	SEGUFIX®-Quick Strap Hand, reinforced	55,40	95°C
4114/r - M	SEGUFIX®-Quick Strap Hand, reinforced	55,40	95°C
4114/r - L	SEGUFIX®-Quick Strap Hand, reinforced	69,20	95°C
4114/re - S	SEGUFIX®-Quick Strap Hand, reinforced, extended	55,40	95°C
4114/re - M	SEGUFIX®-Quick Strap Hand, reinforced, extended	55,40	95°C
4114/re - L	SEGUFIX®-Quick Strap Hand, reinforced, extended	69,20	95°C
4115/r - S	SEGUFIX®-Quick Strap Foot, reinforced	55,40	95°C
4115/r - M	SEGUFIX®-Quick Strap Foot, reinforced	55,40	95°C
4115/r - L	SEGUFIX®-Quick Strap Foot, reinforced	69,20	95°C
4115/re - S	SEGUFIX®-Quick Strap Foot, reinforced, extended	55,40	95°C
4115/re - M	SEGUFIX®-Quick Strap Foot, reinforced, extended	55,40	95°C
4115/re - L	SEGUFIX®-Quick Strap Foot, reinforced, extended	69,20	95°C


*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included


r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

SEGUFIX®-Stretcher Restraint / Mobile Restraint (see page 28 - 31)

Item no.	Product name	Price/€	
5101/r - XS	SEGUFIX®-Restraint Belt, reinforced	120,80	95°C
5101/r - M	SEGUFIX®-Restraint Belt, reinforced	141,20	95°C
5101/r - L	SEGUFIX®-Restraint Belt, reinforced	156,80	95°C
5101/r - XL	SEGUFIX®-Restraint Belt, reinforced	200,40	95°C
5104/r - XS	SEGUFIX®-Hand Straps for Restraint Belt, reinforced	14,60	95°C
5104/r - M	SEGUFIX®-Hand Straps for Restraint Belt, reinforced	15,90	95°C
5104/r - L	SEGUFIX®-Hand Straps for Restraint Belt, reinforced	19,00	95°C
5104/r - XL	SEGUFIX®-Hand Straps for Restraint Belt, reinforced	23,70	95°C
5118	SEGUFIX®-Restraining Aid	30,70	95°C
5143 - S	SEGUFIX®-Stretcher Strap Shoulder	76,40	95°C
5143 - M	SEGUFIX®-Stretcher Strap Shoulder	76,40	95°C
5143 - L	SEGUFIX®-Stretcher Strap Shoulder	81,60	95°C
5145 - M	SEGUFIX®-Stretcher Strap Foot	68,80	95°C

SEGUFIX®-Transport System (see page 32 - 35)

Item no.	Product name	Total length*	Price/€	
6040	SEGUFIX®-Crotch Pad	—	29,50	30°C
6116/v - S	SEGUFIX®-Transport Waist Belt	—	37,30	30°C
6116/v - M	SEGUFIX®-Transport Waist Belt	—	40,40	30°C
6116/v - L	SEGUFIX®-Transport Waist Belt	—	43,50	30°C
6140 - S	SEGUFIX®-Support Belt	—	120,60	30°C
6140 - M	SEGUFIX®-Support Belt	—	127,60	30°C
6140 - L	SEGUFIX®-Support Belt	—	131,50	30°C
6156/v - XS	SEGUFIX®-Multi Belt	160 cm / 63.0 inches	44,90	30°C
6156/v - S	SEGUFIX®-Multi Belt	180 cm / 70.9 inches	44,90	30°C
6156/v - M	SEGUFIX®-Multi Belt	200 cm / 78.7 inches	53,10	30°C
6156/v - L	SEGUFIX®-Multi Belt	240 cm / 94.5 inches	53,10	30°C
6166	SEGUFIX®-Pelvic Belt	—	46,90	30°C
6176	SEGUFIX®-Pelvic Belt with Crotch Strap	—	59,40	30°C


*approximate measurements - products are not pre-washed - about 8 % shrinkage possible

Recommended retail price in €
sales tax not included

r = reinforced
f = fleece

e = extended bed belts
v = Velcro® fastener

SEGUFIX®-Chair Belt System (see page 36 - 39)

Item no.	Product name	Price/€	
7140 - S	SEGUFIX®-Chair Belt, Snap Lock and Magnetic Lock	69,40	95°C
7140 - M	SEGUFIX®-Chair Belt, Snap Lock and Magnetic Lock	75,60	95°C
7140 - L	SEGUFIX®-Chair Belt, Snap Lock and Magnetic Lock	83,30	95°C
7140/f - S	SEGUFIX®-Chair Belt Fleece, Snap Lock and Magnetic Lock	103,70	95°C
7140/f - M	SEGUFIX®-Chair Belt Fleece, Snap Lock and Magnetic Lock	110,50	95°C
7140/f - L	SEGUFIX®-Chair Belt Fleece, Snap Lock and Magnetic Lock	127,00	95°C
7146 - S	SEGUFIX®-Chair Belt, Magnetic Lock	82,40	95°C
7146 - M	SEGUFIX®-Chair Belt, Magnetic Lock	89,50	95°C
7146 - L	SEGUFIX®-Chair Belt, Magnetic Lock	101,80	95°C
7146/f - S	SEGUFIX®-Chair Belt Fleece, Magnetic Lock	116,70	95°C
7146/f - M	SEGUFIX®-Chair Belt Fleece, Magnetic Lock	124,70	95°C
7146/f - L	SEGUFIX®-Chair Belt Fleece, Magnetic Lock	145,50	95°C
7150 - S	SEGUFIX®-Chair Belt with Crotch Strap, Snap Lock and Magnetic Lock	104,00	95°C
7150 - M	SEGUFIX®-Chair Belt with Crotch Strap, Snap Lock and Magnetic Lock	110,20	95°C
7150 - L	SEGUFIX®-Chair Belt with Crotch Strap, Snap Lock and Magnetic Lock	120,90	95°C
7150/f - S	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Snap Lock and Magnetic Lock	138,30	95°C
7150/f - M	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Snap Lock and Magnetic Lock	145,10	95°C
7150/f - L	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Snap Lock and Magnetic Lock	164,60	95°C
7156 - S	SEGUFIX®-Chair Belt with Crotch Strap, Magnetic Lock	117,00	95°C
7156 - M	SEGUFIX®-Chair Belt with Crotch Strap, Magnetic Lock	124,10	95°C
7156 - L	SEGUFIX®-Chair Belt with Crotch Strap, Magnetic Lock	136,40	95°C
7156/f - S	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Magnetic Lock	151,30	95°C
7156/f - M	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Magnetic Lock	159,00	95°C
7156/f - L	SEGUFIX®-Chair Belt with Crotch Strap Fleece, Magnetic Lock	180,10	95°C
7160 - S	SEGUFIX®-Chair Belt with Thigh Straps, Snap Lock and Magnetic Lock	130,90	95°C
7160 - M	SEGUFIX®-Chair Belt with Thigh Straps, Snap Lock and Magnetic Lock	140,80	95°C
7160 - L	SEGUFIX®-Chair Belt with Thigh Straps, Snap Lock and Magnetic Lock	151,70	95°C
7160/f - S	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Snap Lock and Magnetic Lock	165,20	95°C
7160/f - M	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Snap Lock and Magnetic Lock	175,70	95°C
7160/f - L	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Snap Lock and Magnetic Lock	195,40	95°C
7166 - S	SEGUFIX®-Chair Belt with Thigh Straps, Magnetic Lock	143,20	95°C
7166 - M	SEGUFIX®-Chair Belt with Thigh Straps, Magnetic Lock	153,10	95°C
7166 - L	SEGUFIX®-Chair Belt with Thigh Straps, Magnetic Lock	164,00	95°C
7166/f - S	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Magnetic Lock	177,50	95°C
7166/f - M	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Magnetic Lock	188,00	95°C
7166/f - L	SEGUFIX®-Chair Belt with Thigh Straps Fleece, Magnetic Lock	207,70	95°C
7180	SEGUFIX®-Foot Strap Transport	24,60	95°C

Locks & Keys
Item no. 1106 - 1215

Bed Restraint System
Item no. 2203 - 2250

Auxiliary Belts
Item no. 3110 - 3245

Quick Restraint
Item no. 4213 - 4235

Mobile Restraint
Item no. 5204 - 5245

Transport System
Item no. 6040 - 6276

Chair Belt System
Item no. 7240 - 7280

More Products
Item no. 8010 - 8022

General Terms and Conditions

1. Validity of these terms and conditions

The following terms and conditions apply to all transactions of SEGUFIX® GmbH & Co. KG, referred to below as the Supplier, unless other terms are specified in the quote or in the order confirmation of the Supplier. Any additional terms that deviate from these, as well as subsidiary agreements are only binding if they have been confirmed in writing by the Supplier.

2. Quote and signing of contract

All agreements must be in written format, whether they are made at the time the contract was signed or later.

3. Scope of delivery and pricing

3.1 The pricing conditions are based essentially on our respectively applicable price lists plus sales tax as required by law. In the case of written order confirmations, the prices listed in our order confirmation plus sales tax as required by law are definitive. Prices that are agreed upon apply only to the accepted order in question.

3.2 Unless some other agreement is made, the prices for deliveries and services are understood as ex works. Additional deliveries and services will be calculated separately. The applicable freight tariffs will be applied to the freight calculation. In particular, tolls, consulate fees and other taxes, levies and fees resulting from requirements outside the Federal Republic of Germany as well as costs associated with them will be paid by the company placing the order. Upon delivery including customs or other levies, the specified prices are based on the rates in effect at the time of the quote. The actual costs will be calculated. Any accruing sales tax will be calculated in addition.

3.3 Partial deliveries are permitted at any time.

4. Terms and conditions of payment

4.1 All payments must be made without discount to one of the bank connections indicated in the invoice immediately after the invoice is presented. Payments will be considered as made on time if the Supplier has access to the amount within the time allotted. At the discretion of the Supplier, payments may be credited to other receivables that have not yet been paid.

4.2 Checks and drafts – insofar as draft payments have been agreed upon – are accepted as payment. Discount and collection fees as well as interest must be paid to the Supplier without delay.

4.3 A payment is late when a reminder is sent after the due date, but in any case 30 days after it is due and an invoice or equivalent request for payment is received. Unless a more serious late status is demonstrated, interest will be charged on the amount due from the time it was due at 5% above the prime lending rate.

4.4 If the company placing an order stops making payments, is involved in excessive indebtedness or files to begin bankruptcy proceedings, or falls in arrears in redeeming drafts or checks that are due, the entire amount receivable for the Supplier becomes due immediately. The same applies to any other significant deterioration in the financial status of the company placing the order. In these cases, the Supplier has the right to withdraw from the contract.

4.5 The company placing the order is not permitted to withhold payments because of deficiencies that are actually or allegedly present if it is a merchant as defined by the German Financial Law Code (Handelsgesetzbuch -HGB).

Other provisions will apply only if the Supplier is guilty of gross negligence, has already received the portion of compensation for its services corresponding to its service or if the counterclaim on which the right of retainer is based is uncontested, has legal force or if due for a decision.

In any case, the Supplier has the right to replace the amount retained by the company placing the order with a guaranty of payment that extends for the period of the warrantee.

4.6 The company placing the order may not make adjustment or withhold payments unless such adjustment or withholding of payments is uncontested or has the force of law. The Supplier has the right to take precautionary measures to prevent the company placing the order from exercising the right to withhold payments, including a security.

4.7 To ensure credit risk, and depending on the credit worthiness of the company placing the order, the Supplier reserves the right to make a delivery requested by the company placing the order only for cash on delivery.

5. Reserved ownership

5.1 The goods remain the property of the Supplier until all claims the Supplier has against the company placing the order are fulfilled (reserved goods), even if an individual item has been paid for. Pawning or transfer of title to the reserved goods is prohibited. If a claim is made to the delivered item by another party, in particular if the item is pawned, the company placing the order is required to notify the Supplier of this immediately.

Violation of this obligation to notify will serve as justification for the Supplier to withdraw from the contract.

5.2 If the company placing the order is late in whole or partially with its requirement to pay or to redeem drafts or checks that are due, if there is excessive indebtedness or failure to pay, or if bankruptcy is filed (protection from creditors), the following subsidiary protection agreements apply: Where this is permitted as part of regular business operations, in the case where reserved goods are resold, the company placing the order transfers to the Supplier, by way of a precautionary measure, future receivables from its customers it will receive from the resale up to and including repayment of all the Supplier's receivable amounts, without any special requirement for a later explanation. The conveyance also extends to residual receivables resulting from existing open account conditions or upon termination of such relationships of the company placing the order with its customers.

If the reserved goods are resold together with other objects, without having agreed upon an individual price for the reserved goods, the company placing the order transfers to the Supplier, with precedence over remaining receivables, that portion of the total receivable price corresponding to the value of the reserved goods invoiced by the Supplier. The company placing the order is authorized for collect receivables from resale to be forwarded to the Supplier until revocation. However, it is not authorized to have access to them in any other way, for example by conveyance. Upon request of the Supplier, the company placing the order must give notification of the conveyance and must hand over documents required to exercise its rights against customers to the Supplier, for example invoices, and to provide necessary information. The company placing the order is responsible for all receiving costs.

If the company placing the order receives drafts because of the authorization granted it to receive receivables from resale for the Supplier, ownership of these papers is transferred to the Supplier with the right of conveyance as a precautionary measure. The transfer of drafts is replaced by the agreement that the company placing the order is accepting them in safekeeping for the Supplier and will then deliver them directly to the Supplier fully endorsed. In the event that the compensation of the receivable forwarded to the Supplier should be received by the company placing the order or by its financial institution as checks, it must report this as soon as such funds are received and remit the amount. Ownership of the checks is transferred to the Supplier with the right of conveyance as soon as the company placing the order receives them. The transfer of papers is replaced by the agreement that the company placing the order is receiving them in safekeeping for the Supplier, and that it will immediately deliver them to the Supplier fully endorsed.

5.3 If the precautionary case described in Section 5.2 occurs, the Supplier furthermore has the right to take possession of all materials still subject to reserved ownership. It may also exercise its further rights of reserved ownership immediately. The same applies in the case of an otherwise significant deterioration in the financial status of the company placing the order. The company placing the order will allow access to all of its business offices to the Supplier or the Supplier's representative during regular business hours. The demand to surrender or take possession of property does not represent withdrawal from the contract. The Supplier has the right to exploit the reserved goods with the normal care of a proper merchant and to derive profit from then to satisfy open claims against them.

6. Delivery time and late delivery

6.1 Delivery times are not binding unless they have been approved in writing by the Supplier.

6.2 Meeting the delivery time requires the company placing the order to fulfill the obligations to cooperate that it has agreed to contractually. In particular, the all details of the order must be settled, all approvals have been issued and all documentation, payments and securities the company placing the order is required to provide have been received on time by the Supplier. Therefore, delivery time does not begin to run until the requirements above have been fulfilled. Delivery time has been met if the shipment is ready for shipping within the time agreed upon and notification to this effect has been sent to the company placing the order.

6.3 If the Supplier is prevented from executing deliveries and services in a timely manner by mobilization, war, civil unrest, a strike, lockout, interruption of operations, fire, natural catastrophe, transportation impediments, a change in legal requirements, governmental measures or regulations or the occurrence of other unforeseeable events beyond the Supplier's control, the delivery time will be lengthened appropriately.

6.4 If the delivery time is delayed by a circumstance for which the Supplier is responsible, the company placing the order may not withdraw from the contract or demand compensation because of failure to perform if it has previously and without success set a subsequent deadline of at least two weeks for the Supplier and threatened to take such action and this time period has expired with no positive result.

If the company placing the order takes advantage of its right to demand compensation because of failure to perform in this case, it will be considered as it would be if it had no trust in the validity of the contract. Other provisions will apply only if the Supplier is responsible of gross negligence.

6.5 Claims for damages suffered by the company placing the order including from late delivery, and in particular for negligent violation of contract, unauthorized actions committed deliberately and resulting damages are excluded. This does not apply in the case of coercive liability for legal reasons for cases of deliberate action or gross negligence, for guaranteed features or violation of significant contractual obligations due to simple negligence for foreseeable damages typical for contracts.

6.6 The company placing the order is responsible for the additional costs of an interruption or delay in work for which the Supplier is responsible.

6.7 If the company placing the order is responsible for the shipment being late, the Supplier has the right to store the delivery object at the risk of the company placing the order and to demand compensation for resulting costs. In this case, the Supplier has the right to take out insurance covering storage risks at the expense of the company placing the order.

6.8 If the company placing the order is responsible for providing transportation materials for the delivery and it does not do so by the time agreed upon contractually, the Supplier is released of its obligation to deliver by storing and ensuring the delivery objects at the cost and risk of the company placing the order. The shipping or acceptance certificate is sufficient to prove delivery in accordance with the contract.

7. Transfer of risk

As a rule, risk is transferred to the company placing the order as soon as the object being delivered leaves the factory or is made available to the company placing the order in the factory. If shipment is delayed for reasons for which the Supplier is not responsible, risk is transferred to the company placing the order when notification of readiness to ship is issued.

8. Warrantee

8.1 If the sale is a commercial transaction for both parties, the company placing the order must examine the materials immediately after delivery and must notify the Supplier without delay if any deficiencies are apparent.

8.2 If the company placing the order fails to make such notification, the materials are considered to be accepted unless the deficiency is one that could not be detected during the examination. In addition, the provisions of §§ 377 ff of HGB apply.

8.3 Warrantee claims are limited to repair or replacement delivery at the discretion of the Supplier. If the first attempt to make repairs fails or if a deficiency occurs again, the Supplier has the right to make a second attempt to repair to the problem or to make a replacement delivery. This does not apply if a further attempt to make repairs or another replacement delivery would be unreasonable for the company placing the order in a particular case. The company placing the order is responsible for explaining and demonstrating this would be unreasonable. The company placing the order is not authorized to demand reduced compensation or cancellation of the contract (at its discretion) until the second attempt to make repairs or the replacement delivery has also failed.

8.4 More extensive claims of the company placing the order, especially due to damages resulting from deficiencies are excluded unless they are the result of a lack of a feature of SEGUFIX®-Locks that was guaranteed. This does not apply in the case of malice, gross negligence or violation of significant contractual obligations on the part of the Supplier.

9. Return of flawless products

Return of products in their original packaging may be applied for. A restocking fee will be charged.

10. Final conditions

10.1 This contract is governed by the law of the Federal Republic of Germany.

10.2 The place of performance all contractual and legal claims is the headquarters of the Supplier.

10.3 If the company placing the order is a merchant, then Jesteburg is the exclusive legal jurisdiction, including check and draft transactions. The same legal jurisdiction applies if the company placing the order has no general place of legal jurisdiction in the Federal Republic of Germany at the time when legal proceedings are initiated. However, the Supplier does have the right to appeal to any court with legal jurisdiction.

10.4 If individual conditions of the contract are unenforceable, the remaining parts of the contract remain valid. If a stipulation is or becomes unenforceable in whole or in part, the contractual parties will immediately endeavor to achieve the intended financial result of the unenforceable stipulation by another legally permitted means.

10.5 The preceding conditions also apply to deliveries outside of Germany.

10.6 All declarations of intention of the Supplier and the company placing the order in these General Terms and Conditions of Business must be in writing.

In case of a divergence caused by translation the German original text will be applied.

Purchase order form. Thank you for your order! Your SEGUFIX®-Team.

SEGUFIX®-Bandagen
Das Humane System GmbH & Co. KG
Allerbeeksring 33
D-21266 Jesteburg / GERMANY

Fax: +49 (0) 4183 / 500-200

Phone: +49 (0) 4183 / 500-0

E-Mail: info@segufix.de

Visit us online at: www.segufix.com
or buy online at: www.segufix-shop.com

Billing address

customer no.: _____

company: _____

type of business: _____

street: _____

postal code / zip code: _____

city / country: _____

Contact information

Mr. ☐ Mrs. ☐

name, surname: _____

titel: _____

phone: _____

fax: _____

e-Mail:

Shipping address

(if different from billing address)

company: _____

street: _____

postal code / zip code: _____

city / country: _____

name, surname: _____

department:

phone: _____

new customer ☐

existing customer ☐

PO-number: _____

commission:

qty.	item no.	item description	unit price	total price
Total:				

plus sales tax and shipping costs

Our general terms and conditions apply (see page 50).





For more than 40 years:

Competence

Experience

Quality

Dependability

Innovation

Edition 9.5 - 2015-02

SEGUFIX®-Bed Restraints and SEGUFIX®-Mobile Restraints are made from a boil washable cotton rayon mix. SEGUFIX®-Transport Belts are made from polyester and some are made from a cotton rayon mix.

Custom made products and product training are available on request.

Recommended retail prices in € (sales tax not included). The general business and delivery terms apply.

Prices and product specifications subject to change.

Deviations from pictures in colour and shape, errors and amendments excepted.

Velcro® is a registered trademark of Velcro® Industries B.V.

SEGUFIX® is a registered trademark of SEGUFIX®-Bandagen, Das Humane System GmbH & Co. KG.

© SEGUFIX®-Bandagen, 2015

All rights reserved. Reprints or distribution using electronic systems/media, in its entirety or in parts, with prior explicit authorization in writing only.

All requests concerning this must be directed at:
SEGUFIX®-Bandagen
Das Humane System GmbH & Co. KG, Allerbeeksring 33,
D-21266 Jesteburg, Germany

SEGUFIX®-Bandagen

Das Humane System GmbH & Co. KG
P.O. Box 1142 · D-21262 Jesteburg
Allerbeeksring 33 · D-21266 Jesteburg
GERMANY

Phone: +49 (0) 4183 500-0
Fax: +49 (0) 4183 500-200
E-mail: info@segufix.de
www.segufix.com
www.segufix-shop.com